## 4R72=32, 4R72=45, 4R72=60, 4R72=75

<b>DE</b>	Gebrauchsanweisung .....	3
<b>EN</b>	Instructions for use .....	8
<b>FR</b>	Instructions d'utilisation .....	14
<b>IT</b>	Istruzioni per l'uso .....	20
<b>ES</b>	Instrucciones de uso .....	26
<b>PT</b>	Manual de utilização .....	32
<b>NL</b>	Gebruiksaanwijzing .....	38
<b>SV</b>	Bruksanvisning .....	44
<b>DA</b>	Brugsanvisning .....	50
<b>NO</b>	Bruksanvisning .....	55
<b>FI</b>	Käyttöohje .....	61
<b>PL</b>	Instrukcja użytkowania .....	67
<b>HU</b>	Használati utasítás .....	73
<b>CS</b>	Návod k použití .....	78
<b>RO</b>	Instrucțiuni de utilizare .....	84
<b>HR</b>	Upute za uporabu .....	90
<b>SL</b>	Navodila za uporabo .....	95
<b>SK</b>	Návod na používanie .....	101
<b>BG</b>	Инструкция за употреба .....	107
<b>TR</b>	Kullanma talimatı .....	113
<b>EL</b>	Οδηγίες χρήσης .....	118
<b>RU</b>	Руководство по применению .....	125
<b>JA</b>	取扱説明書 .....	131
<b>ZH</b>	使用说明书 .....	137
<b>KO</b>	사용 설명서 .....	142

## INFORMATION

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### INFORMATION

Last update: 2015-12-01

- ▶ Please read this document carefully before using the product.
- ▶ Follow the safety instructions to avoid injuries and damage to the product.

- ▶ Instruct the user in the proper and safe use of the product.
- ▶ Please keep this document in a safe place.

## 1.1 Construction and Function

The 4R72=32, 4R72=45, 4R72=60, 4R72=75 Double Adapters are used to connect two pyramid adapters. The pyramid receivers make it possible to make distal and proximal angle adjustments in the frontal and sagittal planes.

## 1.2 Combination possibilities

This prosthetic component is part of the Ottobock modular system. As a result of its design, it may be combined with other prosthetic components of the modular system. You can find possible restrictions in this section. Contact the manufacturer with questions.

## 2 Application

### 2.1 Indications for use

The product is to be used solely for lower limb prosthetic fittings.

### 2.2 Area of application

Approved for a body weight of **up to 150 kg**.

### 2.3 Environmental conditions

#### For Use of 506G3 Set Screws

Allowable environmental conditions
Temperature range for use: -10 °C to +60 °C
Allowable relative humidity 0 % to 90 %, non-condensing
Unallowable environmental conditions
Mechanical vibrations or impacts
Perspiration, urine, fresh water, salt water, acids
Dust, sand, highly hygroscopic particles (e.g. talcum)

#### For Use of 506G5 Set Screws

Allowable environmental conditions
Temperature range for use: -10 °C to +60 °C
Storage and transport conditions: -10 °C to +60 °C, 250 mbar to 1013 mbar, condensing
Acid resistance: salt water, chlorine water, soapsuds, perspiration, urine
Moisture: fresh water, salt water, chlorine water, soapsuds, steam
Allowable immersion depth: 3 m

### Unallowable environmental conditions

Mechanical vibrations or impacts

Dust, sand, highly hygroscopic particles (e.g. talcum)

## 2.4 Service Life

In general, all modular adapters are tested by the manufacturer for 3 million load cycles. Depending on the patient's level of activity, this corresponds to a service life of 3 to 5 years.

## 3 Safety

### 3.1 Explanation of warning symbols

 **CAUTION** Warning regarding possible risks of accident or injury.

 **NOTICE** Warning regarding possible technical damage.

### 3.2 General safety instructions

#### **CAUTION**

#### **Unallowable combination of prosthetic components**

Risk of injury due to breakage or deformation of the product

- ▶ Only combine the product with prosthetic components that are approved according to the section "Combination possibilities".
- ▶ Based on the instructions for use of the prosthetic components, verify that they may be combined with each other.

#### **CAUTION**

#### **Excessive strain on the product**

Risk of injury due to breakage of load-bearing components

- ▶ Use the prosthetic components according to the classification (see section "Area of application").

#### **CAUTION**

#### **Exceeding the service life**

Risk of injury due to change in or loss of functionality and damage to the product

- ▶ Ensure that the approved service life is not exceeded (see section "Service life").

**⚠ CAUTION****Mechanical damage to the product**

Risk of injury due to change in or loss of functionality

- ▶ Use caution when working with the product.
- ▶ If the product is damaged, check it for proper function and readiness for use.
- ▶ In case of changes in or loss of functionality, do not continue using the product (see "Signs of changes in or loss of functionality during use" in this section).
- ▶ Take any necessary measures (e.g. repair, replacement, inspection by the manufacturer's customer service, etc.).

**NOTICE****Use under unallowable environmental conditions**

Damage to product due to unallowable environmental conditions

- ▶ Do not expose the product to unallowable environmental conditions (see the section "Environmental Conditions").
- ▶ If the product has been exposed to unallowable environmental conditions, check it for damage.
- ▶ If damage is apparent or in case of doubt, do not continue using the product.
- ▶ Take suitable measures if required (e. g. cleaning, repair, replacement, inspection by the manufacturer or a specialist workshop etc.).

**Signs of changes in or loss of functionality during use**

Among other factors, changes in functionality can be indicated by an altered gait pattern, a change in the positioning of the prosthetic components relative to each other and by the development of noises.

**4 Scope of delivery**

4R72=*		
Quantity	Designation	Reference number
1	Instructions for use	647H34
1	Double adapter	-
8	Set screw	506G3=M8x12-V

## 5 Preparation for use

### CAUTION

#### **Incorrect alignment or assembly**

Risk of injury due to damaged prosthetic components

- ▶ Observe the alignment and assembly instructions.

### INFORMATION

For a waterproof prosthesis, use titanium screws and set screws.

## 5.1 Installation in the modular prosthesis

### CAUTION

#### **Improper assembly of the screw connections**

Risk of injury due to breakage or loosening of the screw connections

- ▶ Clean the threads before every installation.
- ▶ Apply the specified torque values.
- ▶ Follow the instructions regarding the length of the screws and about how to secure the screws.

The pyramid adapter is fixed with the set screws of the pyramid receiver.

#### > **Required tools and materials:**

710D4 torque wrench, 636K13 Loctite®

#### 1) **Trial fitting:**

Screw in the set screws.

Use the torque wrench to tighten the set screws (**10 Nm**).

#### 2) **Definitive mounting:**

Use Loctite® to secure the set screws.

Screw in the set screws.

Pre-tighten the set screws with the torque wrench (**10 Nm**) and then tighten them (**15 Nm**).

- #### 3) Replace any set screws that are protruding or recessed too much with suitable ones (see selection table).

**Selection table for set screws**

Reference number	Reference number (titanium)	Length (mm)
506G3=M8X12-V	506G5=M8X12	12
506G3=M8X14	506G5=M8X14	14
506G3=M8X16	506G5=M8X16	16

## **Alignment**

The set screws in the pyramid receiver can be used to make static adjustments during alignment, trial fittings and after the prosthesis is finished.

## **Replacement and disassembly**

The set position of the prosthetic component can be maintained during replacement or disassembly. In order to do this, unscrew the two set screws that are screwed in the furthest and located next to each other.

## **6 Cleaning**

- 1) Clean the product with a damp, soft cloth.
- 2) Dry the product with a soft cloth.
- 3) Allow to air dry in order to remove residual moisture.

## **7 Maintenance**

- ▶ The prosthetic components should be inspected after the first 30 days of use.
- ▶ Inspect the entire prosthesis for wear during normal consultations.
- ▶ Conduct annual safety inspections.

## **8 Disposal**

This product may not be disposed of with regular domestic waste in all jurisdictions. Disposal that is not in accordance with the regulations of the country where the product is used may have a detrimental impact on health and the environment. Please observe the information provided by the responsible authorities in the country of use regarding return, collection and disposal procedures.

## **9 Legal information**

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

### **9.1 Liability**

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregard of this document, particularly due to improper use or unauthorised modification of the product.

### **9.2 CE Conformity**

This product meets the requirements of the European Directive 93/42/EEC for medical devices. This product has been classified as a class I device

according to the classification criteria outlined in Annex IX of the directive. The declaration of conformity was therefore created by the manufacturer with sole responsibility according to Annex VII of the directive.

### 9.3 Warranty

The manufacturer warrants this device from the date of purchase. The warranty covers defects that can be proven to be a direct result of flaws in the material, production or construction and that are reported to the manufacturer within the warranty period.

Further information on the warranty terms and conditions can be obtained from the competent manufacturer distribution company.

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Otto Bock HealthCare GmbH