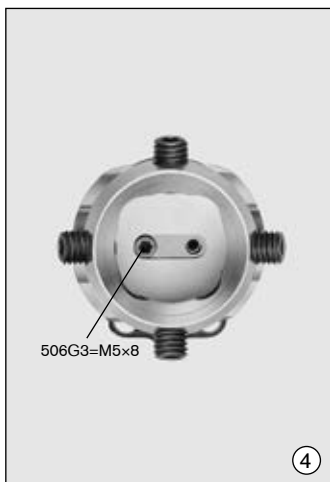
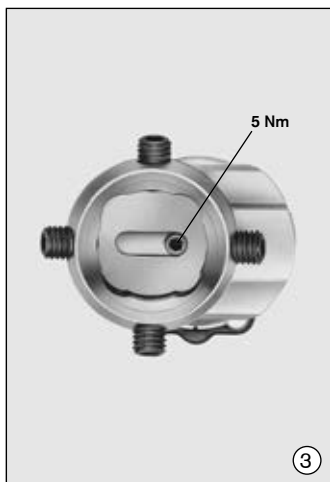
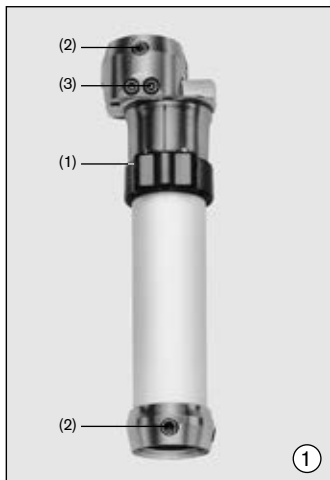
4R88

DE	Gebrauchsanweisung	3
EN	Instructions for use	7
FR	Instructions d'utilisation	11
IT	Istruzioni per l'uso	15
ES	Instrucciones de uso	19
PT	Manual de utilização.....	23
NL	Gebruiksaanwijzing.....	27
SV	Bruksanvisning.....	31
CS	Návod k použití.....	35
TR	Kullanma talimatı.....	39
EL	Οδηγίες χρήσης	43

Achtung!

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Attention!

Last update: 2016-02-25

- Please read this document carefully before using the product.
- Follow the safety instructions to avoid injuries and damage to the product.
- Instruct the user in the proper and safe use of the product.
- Please keep this document in a safe place.

The 4R88 Sliding Adapter is to be **exclusively** used for the prosthetic fitting of amputations of the lower limb.

Field of application according to the **Ottobock Mobility System MOBIS:**

Approved for a patient with a **weight of up to 100 kg/220 lbs.**

The 4R88 Sliding Adapter was designed for modular lower extremity prostheses with 34 mm diameter tube connection. The adapter permits additional sliding adjustments between the prosthetic socket and the tube adapter, even as the prosthesis is being worn. These fine turning adjustments allow a parallel shifting of the distal portion independent of the angle of adjustment through the pyramid adapters. The prosthetic foot may be shifted medially or laterally in the frontal plane, or dorsally or ventrally in the sagittal plane.

**Attention!**

Please avoid exposing prosthetic components to surroundings that corrode metal parts, for example, freshwater, saltwater, acids and other liquids. Using this medical device in such environmental conditions will render all claims against Otto Bock HealthCare null and void.

Please inform your patients.

2 Connecting with the tube adapter and handling of the set screws.

The tube of the 2R57 tube adapter must be thoroughly deburred inside and outside.

**Caution!**

Prior to final assembly (i.e. prior to inserting the tube adapter into the sliding adapter), the inside of the sliding adapter and the outside part of the tube adapter which slides into the sliding adapter must be cleaned with a degreasing cleaning agent (e.g. 634A3 acetone)!

Prior to setting the tube, loosen the 501Z2=M4×22 Cap Screws (Fig. 1, item 1) on the sliding adapter by two turns. Insert the tube up to the stop so that it butts up against the bottom of the clamp on the sliding adapter.

**Attention!**

For final use, the Cap Screw must be torqued to 5 Nm with the 710D122 Torque Wrench. Failing to comply with this instruction may lead to permanent over-extension or breaking of the screw.

Adjustments may be made through adjusting the four 506G3 Adjustment Set Screws (Fig. 1, item 2) during alignment, trial fitting and at any time after the prosthesis has been completed. The optimal position of the components may be preserved when exchanging parts or when servicing the prosthesis by removing only the two most deeply set adjacent screws.

Once positioning has been determined, replace any set screws that are too long (to prevent foam cover damage) or too short (lacking strength) (Fig. 2).

Screws are available in the following sizes:

<i>Article no.</i>	<i>Length</i>
506G3=M8×14	14 mm
506G3=M8×16	16 mm

Use a 710D4 Torque Wrench to tighten screws.

Attention!

For trial fitting: Tighten the 506G3 Set Screws clockwise to **10 Nm (7.4 lb / ft, 88 lb / in)**.

For completion: Secure the 506G3 Set Screws with 636K13 Loctite and pre-tighten them clockwise to **10 Nm (7.4 lb / ft, 88 lb / in)**; then complete assembly by tightening them to **15 Nm (11 lb / ft, 133 lb / in)**.

3 Technical data

Article no.	4R88
Weight	185 g
System height	51 mm
Material	Titanium
Max. patient weight	100 kg
Mobility grade	–

4 Handling

The translation direction is determined by the position of the sliding adapter on the modular tube – in most cases this will be the frontal plane.

The upper section of the adapter may be slid ± 11 mm over the stable dovetail guide and subsequently secured by the 501Z16 Locking Bolts (Fig. 1, item 3).

Torque moment: 15 Nm (11 lbf.ft., 133 lbf.in.).

Use the 710D4 Torque Wrench.

Secure the adjustment screws in their final adjustment position using 636K13 Loctite.

The sliding adjustment is limited by the 506G3=M5x8 Stop Screw (Fig. 3 and 4). For adjustments in the opposite sliding direction, it must be moved.

Attention!

It is not permissible to use the sliding adapter without the 506G3=M5x8 Stop Screw, since this could make the adapter come loose and as a result cause the patient to fall!

Tighten the stop screw using the 710D122 Torque Wrench.
Torque moment: **5 Nm (3.7 lbf.ft., 44 lbf.in.)**.

4.1 Maintenance instructions

Note:

As a basic principle, all Ottobock modular adapters are subjected to tests involving three million load cycles. Depending on the amputee's activity this corresponds to a service life of three to five years.

We recommend carrying out regular safety checks once a year.

5 Legal information

All legal conditions are subject to the respective national laws of the country of use and may vary accordingly.

5.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document. The manufacturer will not assume liability for damage caused by disregard of this document, particularly due to improper use or unauthorised modification of the product.

5.2 CE conformity

This product meets the requirements of the European Directive 93/42/EEC for medical devices. This product has been classified as a class I device according to the classification criteria outlined in Annex IX of the directive. The declaration of conformity was therefore created by the manufacturer with sole responsibility according to Annex VII of the directive.

5.3 Warranty

The manufacturer warrants this device from the date of purchase. The warranty covers defects that can be proven to be a direct result of flaws in the material, production or construction and that are reported to the manufacturer within the warranty period.

Further information on the warranty terms and conditions can be obtained from the competent manufacturer distribution company.

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Otto Bock HealthCare GmbH