

Use and Care

Model # 9830

Clear Shell Helmet


Professionals working with surgery patients may want to consider this helmet as a recovery option. The clear polycarbonate shell is ventilated and edged with soft foam. Available in one size with 8 positioning pads included that can be used to adjust sizing and elevate specific areas of the helmet away from wound sites. Suggestions for padding placement are included with the helmet. Available with pads attached or unattached; comes with an easily adjustable chin strap.



Pads Attached




Pads Unattached

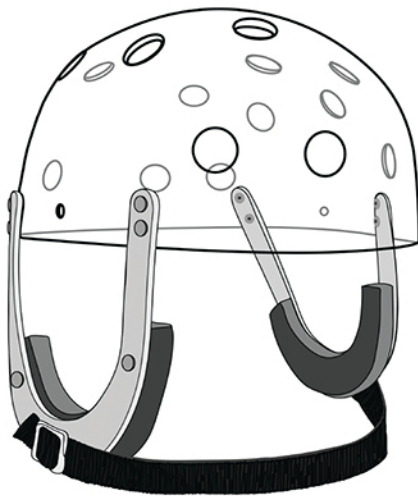

A: Clear Shell Helmet prior to fitting with adhesive pads.

C: The Clear Helmet positioned on wearer's head. Additional pads may be added in open spaces to prevent contact between head and helmet shell. Size may be reduced by applying pads over pads to increase thickness.

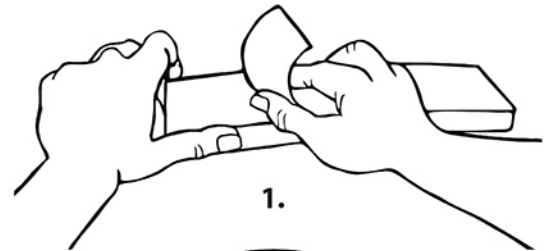

1.



2.

B: To add custom pads

- 1) Remove protective paper backing.
- 2) Insert into helmet, position as needed and press into place.

TEST PAD POSITIONING IN HELMET BEFORE ADHERING AS PADS ARE NOT MEANT TO BE REPOSITIONED AND ARE VERY DIFFICULT TO REMOVE.

