# amoena

## BREAST FORMS & SHAPERS

CONTACT | ADAPT AIR |

BALANCE CONTACT | BALANCE ADAPT AIR

Gebrauchsanweisung. Instructions for use. Mode d'emploi. Instrucciones de uso. Istruzioni per l'uso. Gebruiksaanwijzing. Brugsanvisning. Instruksjoner for bruk. Användningsinstruktioner. Instrukcja użycia. Instrução de uso. Käyttöohjeet. Návod k použití. Инструкции по применению. Οδηγίες χρήσης. 使用説明. 使用説明書.

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**Thank you for choosing a quality product from Amoena Medizin-Orthopädie-Technik GmbH. For more than 45 years, Amoena has been synonymous with innovative and high-quality products that support the needs of women following breast surgery.**

**Intended Purpose/Indication**

Target group: Women after various breast surgeries

Breast forms are required after partial or complete removal of the breast after breast cancer or other diseases and in the case of breast malformation or degeneration. As well as providing a balanced look, the breast form is designed to maintain the symmetry of the body (preventing shoulder misalignment) by ensuring the breast form is the appropriate weight, and to simulate the movement of the natural breast.

Breast forms and breast shapers should only be worn once the scar area has fully healed.

**Contraindications/Risks**

No contraindications or risks are known or expected when using the above-mentioned products. In rare cases allergic reactions may occur.

A contraindication exists if there is a known allergy to one of the components that may come into contact with the skin.

**Material Composition**

- Polyurethane film
- Polydimethylsiloxane and silicone oil
- Also, depending on the item: lightweight microspheres, temperature-regulating material (PCM Phase Change Material), additives and colour pigments

**General Instructions for Use**

- We recommend wearing the breast forms in the pocket of a well-fitting specialist bra. Contact breast forms and breast shapers adhere directly to the skin.
- The breast forms and breast shapers tolerate chlorine and salt water without any problems. However, they can pick up odours over time (e.g. your body odour or odours of external origin such as chlorinated water), which can last for an indefinite period of time. We therefore recommend the Amoena Aqua Wave for water sports activities.
- We recommend that you do not wear breast forms or breast shapers in the sauna or steam bath.

### General Care Instructions

- For the care of your Amoena breast form, we recommend daily washing by hand with the Amoena Soft Cleanser, a mild soap or liquid detergent. It should then be rinsed in clean water and dried with a soft towel. Body lotion, powder and natural body oils, can cause a film to form on the inside of the breast form.
- Please handle your breast form with care. Pointed or sharp items such as pins, animal claws, nail files, scissors, etc. can damage the breast form. Breast forms made of lightweight silicone should be handled with particular care in order to avoid damaging the lightweight silicone. Please do not press these with your fingers and/or nails, just as you would not do with your skin.
- Discolouration does not impair either the quality or function of the breast form.
- Some substances can damage the film coating of the breast form:
  - Alcohols such as ethanol, propanol or butanol when used at the same time as organic, low-molecular substances such as insect repellent (DEET) or sun cream
  - Ethyl acetate and other esters
  - Acetone

Wait until the substances have been absorbed and wash your hands before touching your breast form. Disinfectants are permitted.

### PRODUCT-SPECIFIC NOTES

#### Contact Breast Forms

Art. No.	Article
380	Contact Light 2S
381	Contact 2S
382	Contact 3S
383	Contact 2A
384	Contact 1S
385	Contact Light 3S
386	Contact 3E

All Contact products are delivered with a sample size Amoena Soft Cleanser and a soft brush.

#### Instructions for Use

- The adhesive strength of the breast form can vary depending on your skin type (oily or dry). It is also dependent on how much you sweat, your level of physical exertion and the condition of the scar area.
- However, selective adhesion is sufficient to reduce strain on your shoulder and neck.
- The Contact breast form can be worn when playing sport, but adhesion may be reduced due to the build-up of sweat. The adhesive breast form is chlorine-resistant and salt-resistant. However, it should be cleaned directly after use in clean water with the Amoena Soft Cleanser.
- Wear the Amoena Contact every day from the beginning. The more regularly the breast form is worn, the better your skin will accept the adhesive.
- In the case of extensive and intensive wear, sweat and small skin particles will build up on the adhesive surface in the course of the day. This means adhesion will slowly decrease.
- Thorough cleaning of the adhesive surface before putting it back on will give you a renewed feeling of security.

#### Balance Contact Shapers

Art. No.	Article
229	Balance Contact MD
230	Balance Contact VD
286	Balance Contact SV

- If your skin tends towards dryness, apply moisturising skin care after wearing the Contact breast form.
- For each Contact breast form and Contact breast shaper, a 3-month exchange right applies. If you struggle to achieve adhesion within the first 3 months, you can exchange it for a bra-supported Amoena product.

### Putting On the Contact Breast Form

1. First clean your skin carefully. Then let the skin dry fully. The drier and cleaner your skin is, the better the adhesion of the breast form. Please do not touch the adhesive surface of the breast form, as this can impair the adhesion.
2. Now put on your bra and check the fit. Next pull your bra strap completely off your shoulder and arm and lean forwards.
3. Using the hand from your non-operated side, take hold of the bra strap and pull the cup away from the body.
4. Take the breast form in the other hand and place it in the cup of the bra. The upper corner of the breast form should be close to the bra strap. Stay leaning forwards. Make sure that the breast form sits well in the bra. See Diagram 1 page 3
5. Cup the outer edge of your bra with your free hand and slowly straighten up. As you do so, carefully press the breast form onto your body.
6. Now pull the bra strap over your shoulder and press the breast form firmly into place again. Diagram 2 page 3
7. Check the fit. If the breast form is not well positioned, lean forwards and start over.

### Taking Off the Contact Breast Form

1. Cup the breast form with one hand as completely as possible, so that your fingertips are near the edge, and lean forwards until you feel the weight of the breast form in your hand. Starting at a point some distance from the scar, slowly pull the breast form off the body.
2. If the adhesive surfaces accidentally stick together as they are removed, they can be easily separated again under running water.
3. Please be careful with long fingernails. These can damage the coating of the breast form.

### General Care Instructions

- Skin texture and skin evaporations caused by e.g. medications and care products can affect the adhesive strength. Please clean your breast form daily and thoroughly to achieve optimal and long-term adhesion.

- Only Amoena Soft Cleanser allows you to achieve optimum adhesion. Other cleaning products can reduce the adhesive strength.
- Small clumps that form on the adhesive surface should not be removed. Do not use any other cleaning agents such as abrasive powder or implements such as scissors or tweezers. Clean the adhesive surface with the Amoena Soft Cleanser.
- The special brush should be replaced after 12 months at the latest.

### Cleaning the Adhesive Surface

1. Add three drops of the Amoena Soft Cleanser to the dry adhesive surface and then moisten the special brush with lukewarm water. See Diagram 3 Contact page 3
2. Using only the special brush, clean the adhesive surface well for approx. 3 minutes, applying pressure and using circular movements. See Diagram 4 Contact page 3
3. Then rinse with plenty of lukewarm water.
4. Leave to air dry or dry with a hair dryer on a moderate setting.

### ADAPT BREAST FORMS

Art. No.	Article
326	Adapt Air Xtra Light 2SN
327	Adapt Air Light 2SN
328	Adapt Air Xtra Light 1SN
329	Adapt Air Light 1SN
376	Adapt Light 3A

All Adapt Air products are supplied with a pump.

### BALANCE ADAPT SHAPERS

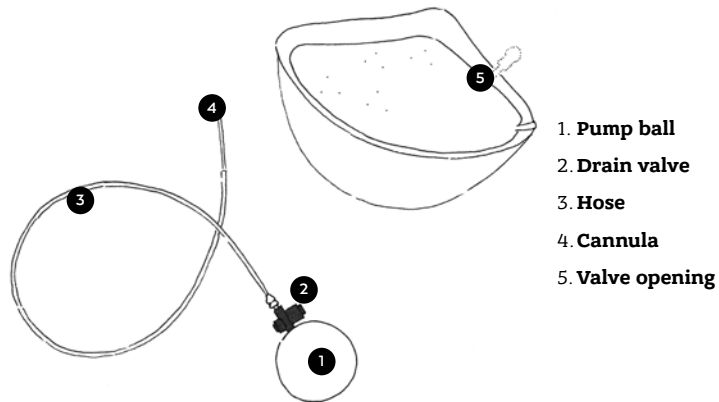
Art. No.	Article
233	Balance Adapt Air MD
234	Balance Adapt Air SV

### General Instructions for Use

Amoena Adapt Air are breast forms with an integrated air chamber which can be individually adjusted to fit the chest wall and the body's natural silhouette.

A special pump is required to fill the chamber with air or drain the air from the chamber. Small quantities of silicone oil, which can form on the cannula of the pump, do not impair the function and quality of the breast form or pump. Simply wipe away the oil with a clean cloth.

### Adjusting Adapt Air

1. Lay out the breast form and pump ready for use. Breast form and pump are separate. Diagram 5 page 3
2. Place the pump (1) under the bra strap on the opposite side so that you have both hands free. Pull the bra strap off the shoulder of the operated side. Diagram 6 page 3
3. Connect the pump by removing the grey tag (if present) and inserting the cannula (4) into the opening (5) of the breast form marked with a white arrow. Open the valve inlet by gently squeezing your thumb and index finger together. Guide the cannula completely into the valve. Diagram 7 page 4

4. Put the breast form into the bra cup. Ensure that the hose is connected correctly with the breast form. Diagram 8 page 4
5. Bring the bra strap back into the correct position. Diagram 9 page 4
6. Now, adjust the breast form to your needs by pressing the pump ball (1). To drain the air, press the black button on the drain valve (2) and lightly press the breast form against the chest wall. Diagram 10 page 4
7. Please note: In order to insert or drain air, the cannula must be inserted completely into the valve.
8. As soon as the correct volume is set, remove the cannula carefully from the valve. The air remains in the air chamber. Diagram 11 page 4
9. Now the breast form has been adapted to your individual needs. Take it out of the bra cup and put it into the bra's breast form pocket. Diagram 12 page 4.

### General Care Instructions for Adapt Air Breast Forms

- When cleaning, ensure that no water enters the valve.
- Due to the valve, we do not recommend you wear Adapt Air when swimming.
- Changes in air pressure can lead to minimal, invisible changes in volume. Adjust the volume with the pump if necessary.
- Use only the pump supplied with the breast form for any adjustments.

### General Care Instructions for Pump

Clean the pump ball, the hose if necessary, and the drain valve with a mild soap solution. Disinfectants which contain alcohol can damage the pump.

The proper function of the pump is guaranteed between -10°C and + 40°C, while the maximum permissible temperature lies between -30°C and +100°C.

It can be disinfected by boiling.

In this case, separate the component parts from the pump ball and boil it in ample water for a short time.

### Pump Materials

Hollow body: Soft PVC, phthalate-free

Silicone hose: Silicone

PP tip: PP

Pressure drain valve: ABS

**Hazard Warning**

Please keep away from children. The product may contain small parts such as adapters or valves. If these parts are swallowed, there is a risk of suffocation.

**Special Instructions**

Protect from sunlight

Store in a dry place

Single patient – multiple uses

**Disposal Instructions**

Dispose of the products in household waste.

**Reporting Obligation**

If you have any complaints about the product, please contact your medical retailer directly. It is only necessary to notify the manufacturer and the competent authority of the member state in the event of any serious incidents that could lead to a significant deterioration of health or to death. Serious incidents are defined in Article 2 No. 65 of Regulation (EU) 2017/745 (MDR).

**Warranty**

The statutory warranty obligation applies.

**Service Information**

Do you have any questions?

If so, we would like to help – please call us or write to us. You can find our address on page 143.

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Amoena is a registered trademark of Amoena Medizin-Orthopädie-Technik GmbH.
© 2021 All rights reserved Amoena Medizin-Orthopädie-Technik GmbH, 83064 Raubling, Germany