






Instructions for Use







---




LSF CALF COSMESIS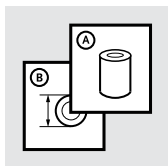

		3
EN	Instructions for Use	6
DE	Gebrauchsanweisung	9
FR	Notice d'utilisation	12
ES	Instrucciones para el uso	15
IT	Istruzioni per l'uso	18
NO	Bruksanvisning	21
DA	Brugsanvisning	24
SV	Bruksanvisning	27
EL	Οδηγίες Χρήσης	30
FI	Käyttöohjeet	33
NL	Gebruiksaanwijzing	36
PT	Instruções de Utilização	39
PL	Instrukcja użytkowania	42
CS	Návod k použití	45
TR	Kullanım Talimatları	48
RU	Инструкция по использованию	51
JA	取扱説明書	54
ZH	中文说明书	57
KO	사용 설명서	59
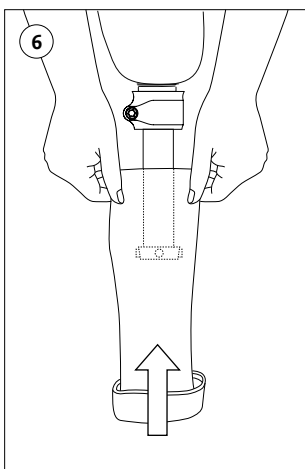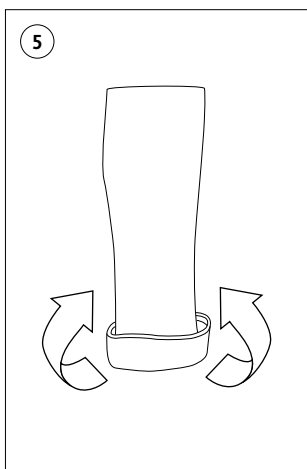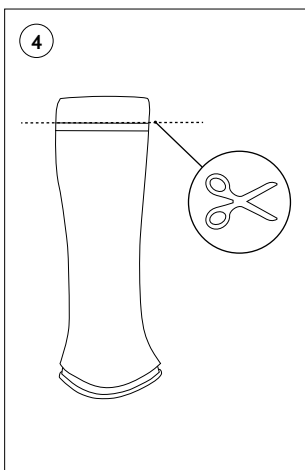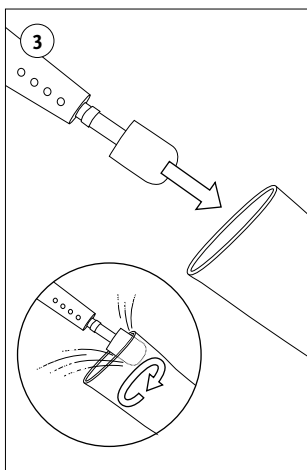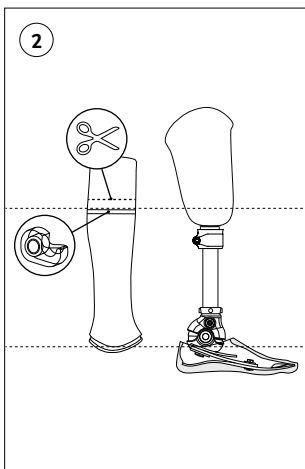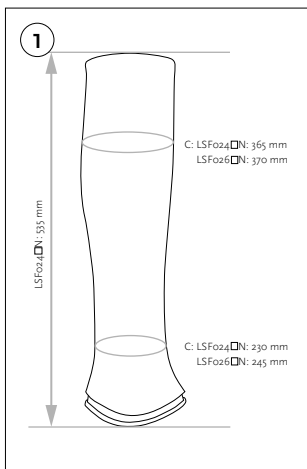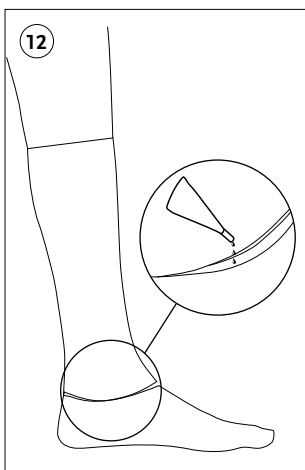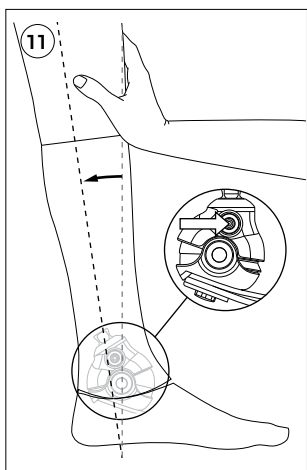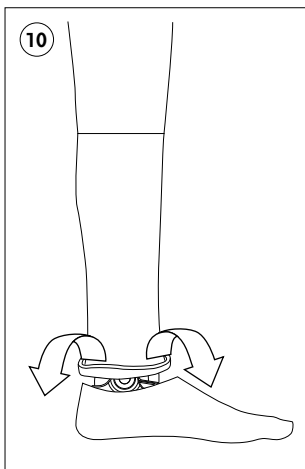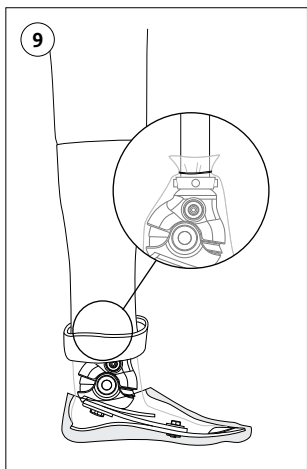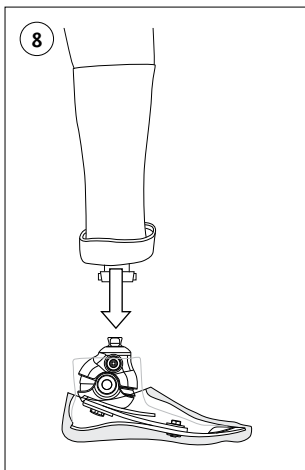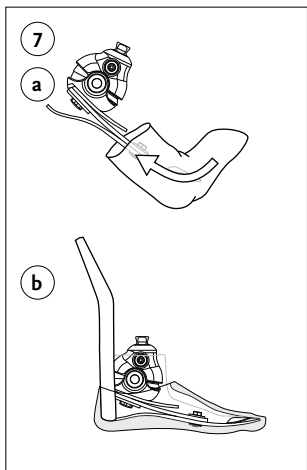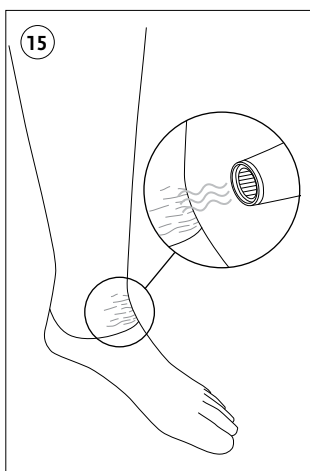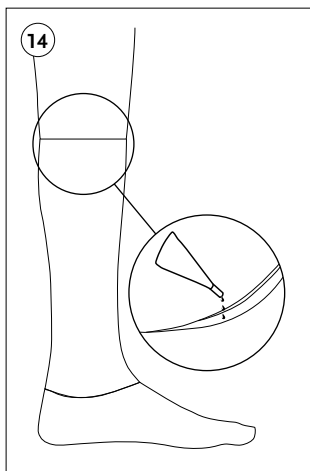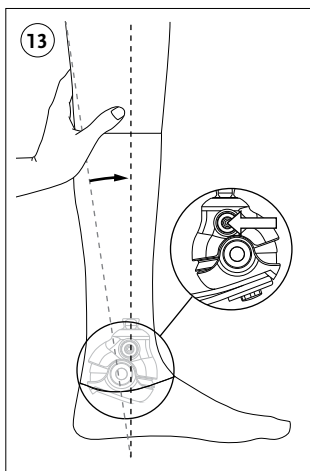
# ENGLISH

---

The **Össur LSF Calf Cosmesis** is referred to as the **device** in the following document. This document provides important information on the indications for use, and handling of the device. It is intended for a certified prosthetist and the user of the device. The device may only be fitted by a qualified practitioner authorized by Össur after completing the corresponding training.

## PRODUCT DESCRIPTION

The device is a cosmetic cover intended to contain the Össur prosthetic leg, providing cosmetic finishing between the prosthetic foot and the socket.

The device is available for right and left sides in two sizes (22-24 cm and 25-26 cm) (see **Figure 1**).

## INDICATIONS FOR USE

The device is a non-invasive, reusable prosthetic device designed for single patient use in low to moderate everyday activities.

## CONTRAINDICATIONS

Not applicable

## TECHNICAL SPECIFICATIONS

The following specifications are for size 22-24 device:

- Weight: 335 g / 0.74 lbs
- System height: 53.5 cm / 21.1" (uncut)
- Circumference:
  - Calf: 36.5 cm / 14.4"
  - Ankle 22.8 cm / 9.0"

## SAFETY

### **Warnings**

The device should only be used with intended Össur products.

The device is weatherproof, therefore cannot be submerged in water (please refer to the *Weatherproof* section).

In cases of change in device performance, damage, or malfunction, the user should immediately cease using the device, and return it to their practitioner.

The device is **NOT** designed for use in extreme sports or other extreme activities.

Exposure to temperatures above 50°C for prolonged periods can result in device damage.

### **Weatherproof**

A weatherproof device allows use in a wet and/or humid environment, but does not allow submersion. Fresh water splashing against the enclosure from any direction shall have no harmful effect. Dry thoroughly after contact with fresh water. Fresh water: Includes tap water. Excludes salt and chlorinated water.

For a weatherproof system, weatherproof components must be selected for use with the device.

## INITIAL FITTING

### Establishing required length (Figure 2)

- Align the device to the user's prosthetic system to measure required length.
- Mark where final upper line of calf fits user best, and mark with tape.
- Cut above taped line to provide an area to hold the device whilst grinding.

### Grinding to fit socket

- Grind the inner surface of the upper device area to fit socket well (Figure 3).
- Components of the prosthetic system can be accommodated by grinding in those areas. Grinding in the ankle area will allow easier access to the Lock/Release button on the prosthetic foot.
- The device can be punctured by an awl or other sharp object to allow easy access to valves or other components.
- After grinding, use pressurized air to remove any debris.

### Cutting to size

- Cut the device at upper line previously marked by tape (Figure 4) for appropriate length, or to mimic socket shape if desired.

### Donning device

- Fold the device up at base (Figure 5).
- Pull device over the prosthetic leg and socket (Figure 6).

### Preparing prosthetic foot

- Place the Spectra sock over the prosthetic foot (Figure 7a), ensuring the sock is pushed into the gap in the sandal toe.
- Don foot cover with a shoe horn (Figure 7b).

### Assembling prosthetic system

- Attach prosthetic socket to foot (Figure 8).
- Tighten bolts to secure.
- Pull Spectra sock up to leg pylon, and fasten with plastic strap (Figure 9).
- Fold device down at base (Figure 10).

### Gluing device to foot cover

- Press the Lock/Release button on the medial side of the prosthetic foot to unlock. Extend the prosthetic system backwards at an angle (Figure 11).
- Place glue (cyanoacrylate or equivalent) around the perimeter of both the device and foot cover (Figure 12).
- Return the prosthetic system to maximum dorsiflexion, and lock the foot by pressing the Lock/Release button on the lateral side (Figure 13).

### Assembly of prosthetic system

- Attach top of device to the socket system (Figure 14).

**NB:** Manipulation of the device during initial fitting may result in wrinkles appearing on the ankle area. In such cases, heat may be applied via a heat gun at 340°C at a length of 10-15 cm for 1-2 minutes, until wrinkles fade (Figure 15).

**MAINTENANCE**

Clean regularly with hygienic wipes, or using a cloth and warm soapy water. For stubborn stains use an alcohol-based solution.

**WARRANTY**

Össur offers a 6-month (from invoice date) limited warranty on the device.

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**EN – Caution:** Össur products and components are designed and tested according to the applicable official standards or an in-house defined standard when no official standard applies. Compatibility and compliance with these standard is achieved only when Össur products are used with other recommended Össur components. If un-usual movement or product wear is detected in a structural part of a device at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist. This product has been designed and tested based on single patient usage. This device should NOT be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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