Explanation of Symbols

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Explication des symboles

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Verklaring van de symbolen

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Application videos are available on www.deroyal.com.

SIZING

Size	Dimensions
Infant	6 ½" L x 7 ½" W (16 x 19 cm)
Pediatric	7" L x 10 ½" W (18 x 25 cm)
Pediatric	13" L x 15" W (33 x 38 cm)

CLEANING AND/OR MAINTENANCE

Prior to washing, remove any detachable rigid stays, fasten any buckles and tie any laces to minimize damage during wash and dry cycles. Close any hook and loop fasteners before washing to help prevent lint build-up and pilling. For maximum product life and to help prevent any straps from tangling, wash in a mesh laundry bag. Hand or machine-wash with a mild fabric detergent in warm water up to 60°C / 140°F. Bleach as directed on container. Do not iron. Lay flat and open and allow to air dry, or tumble dry on low heat for up to 40 minutes. Inspect the product after washing to ensure it is intact and replace any previously removed stays. Hook and loop fasteners may collect lint after repeated use or laundering, reducing grip strength. As needed, use a stiff bristle brush to remove lint from the "hook" side. Test hook and loop adhesion before use and discard product if hook and loop does not hold securely. During the washing, metal buckles could bend and plastic quick-release buckles could crack or break. Always ensure that all buckles are in good working order and are not bent, cracked or broken before each use. If the product is damaged, do not use it and replace it with a new one. Customers could expect up to 5% shrinkage in cotton materials over the life of the product.

WARRANTY

DeRoyal products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

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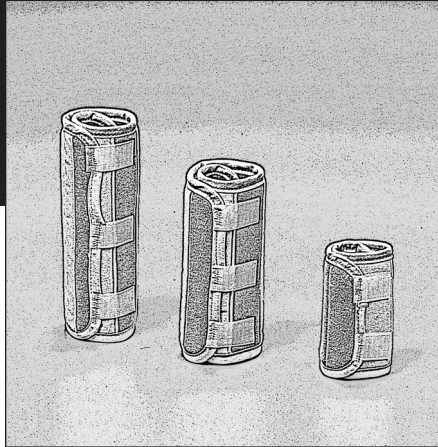
Improving Care. Improving Business.®

	Manufacturer: DeRoyal Industries, Inc. 200 DeBusk Lane Powell, TN 37849 USA 888.938.7828 or 865.938.7828 www.deroyal.com
	QNET BV Hommerterweg 286 6436 AM Amstelveen The Netherlands
	Part#0-1909G Revised 6-2015
	©2015 DeRoyal Industries, Inc. All Rights Reserved.
	CE 6-2015

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PEDIATRIC ELBOW IMMOBILIZER

**RX ONLY
SINGLE PATIENT USE
NON-STERILE
NOT MADE WITH NATURAL RUBBER LATEX**

IMPORTANT INFORMATION

Please read all instructions, warnings, and precautions before use. Correct application is essential for proper functioning of the product.

Use only on the person it was provided to by a healthcare professional and only for the use it was intended.

INTENDED USE

The DeRoyal Elbow Immobilizer is intended to be prescribed by a properly licensed practitioner to provide comfortable support while helping prevent patients from pulling I.V. lines, nasogastric (NG) tubes and tracheostomy tubes.

CONTRAINDICATIONS

- Do not use on a patient who is or may become highly aggressive or agitated.
- Never use on a patient if an I.V. or wound site could be compromised by the device.

WARNINGS

- Improper application or use of any restraint may result in serious injury or death.
- Inspect product for proper fit immediately following patient application.
- Discontinue use of device and consult physician or properly licensed practitioner if device becomes damaged in any way, limiting its ability to perform as intended.
- Inspect product for damaged or missing components prior to patient application.

PRECAUTIONS

This product is to be fitted initially by a physician (or properly licensed practitioner) who is familiar with the purpose for which it is intended. The physician or practitioner is responsible for providing wearing instructions and precautions to other healthcare practitioners or healthcare providers involved in the patient's care as well as the patient. Consult your physician (or properly licensed practitioner) immediately if you experience sensation changes, unusual reactions, swelling or increased pain while using this product. Follow the instructions of your physician (or properly licensed practitioner) for length and duration of use.

Be sure to follow your facility's policies and guidelines for frequency of patient monitoring.

INSTRUCTIONS FOR USE

- Place the immobilizer under the patient's arm and make sure the elbow is placed in the center of the device.
- Wrap the immobilizer around the arm. Start with the middle tie closure, then top and bottom closures. Make sure to use quick release ties.

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