#### FITTING INSTRUCTIONS

1. Position the removable foam insert vertically on the felt pad.
2. Place arms through arm straps so that felt pad is located on back and the foam insert is between shoulder blades.
3. Adjust by pulling the arm straps tight. Fasten the contact closures on the arm pad.
4. The cushioned arm pad should be adjusted so that it fits under the arm and in front of the shoulder. The straps should be tight enough to pull the shoulders back.

#### WASHING INSTRUCTIONS

This product may be cleaned using mild detergent and cold water. Remove foam insert before washing. Do not bleach. Do not wash in washing machine or place in dryer. Lay brace flat on a towel to air dry. Do not squeeze or wring out brace.

#### GENERAL SAFETY INFORMATION

**WARNING:** In cases of functional change or functional loss, the patient should stop using the device and contact a healthcare professional.

**WARNING:** If the device shows signs of damage or wear hindering its normal functions, the patient should stop using the device and contact a healthcare professional.

#### REPORT A SERIOUS INCIDENT

**Important notice to users and/or patients established in Europe:**

The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

#### DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

#### LIABILITY TEXT

Össur does not assume liability for the following:

- Device not maintained as instructed by the instructions for use.
- Device assembled with components from other manufacturers.
- Device used outside of recommended use condition, application, or environment.

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©Copyright Össur 2021 IFU0322 1474\_001 Rev. 2
Össur Deutschland GmbH