

# INSTRUCTION SHEET

## ENDOSKELETAL COMPONENT

### 4-Hole Pyramid Receiver, Ti / Part No. 4HPR T

#### TECHNICAL INFORMATION

<b>Material</b>	Titanium
<b>Build Height</b>	17 mm (0.67 in)
<b>Assembly Weight</b>	64 g
<b>Patient Weight Limit</b>	136 kg (300 lbs)
<b>Warranty</b>	3 Year, limited
<b>Torque (Fasteners)</b>	12 N·m (8.8 ft-lbs)
<b>Torque (Set Screws)</b>	15 N·m (11 ft-lbs)

The M6 x 14mm fasteners included are for use with non-rotatable adapters. Use of longer fasteners may be required for rotatable adapters.

#### FASTENER SELECTION

#### POTENTIAL RESULT

Thread length too short	Inadequate thread engagement
Thread length too long	Bottoming-out or binding

- ⚠ CAUTIONS**
- Apply Loctite® 242® on all M6 x14mm fastener threads.
  - M8 x 14mm set screws do not require Loctite.
  - Torque evenly to specified settings.
  - Do not expose this product to corrosive materials.

*Failure to follow these guidelines may result in one or all of the following: noise, movement, damage, and catastrophic failure. For further questions, please contact College Park Technical Service.*



## college park

800.728.7950 | 586.294.7950 | college-park.com



**COLLEGE PARK INDUSTRIES**  
27955 College Park Dr.  
Warren, MI 48088 USA



**EMERGO EUROPE**  
Prinsessegracht 20, 2514 AP  
The Hague, Netherlands

**MADE IN THE USA**  
225 INS 4HPR 170804



**Australian Sponsor**  
**EMERGO AUSTRALIA**  
Level 20, Tower II, Darling Park,  
201 Sussex Street,  
Sydney, NSW 2000 Australia

©2017 College Park Industries, Inc. All rights reserved. College Park is a registered trademark of College Park Industries, Inc.