dynion explore 3R84=P

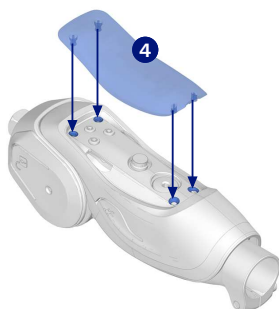
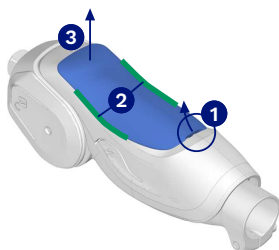
DE	Gebruiksaanweisung (Anwender)	5
EN	Instructions for use (user)	13
FR	Instructions d'utilisation (utilisateur)	21
IT	Istruzioni per l'uso (utente)	29
ES	Instrucciones de uso (usuario)	37
PT	Instruções de utilização (utilizador)	45
NL	Gebruiksaanwijzing (gebruiker)	53
SV	Bruksanvisning (användare)	61
DA	Brugsanvisning (bruger)	69
NO	Bruksanvisning (bruker)	76
FI	Käyttöohje (käyttäjä)	84
PL	Instrukcja użytkowania (użytkownik)	91
HU	Használati útmutató (felhasználó)	99
CS	Návod k použití (uživatel)	107
RO	Instrucţiuni de utilizare (utilizator)	115
HR	Upute za uporabu (korisnik)	123
SL	Navodila za uporabo (uporabnik)	131
SK	Návod na použitie (používateľ)	138
BG	Инструкция за употреба (потребители)	146
TR	Kullanım kılavuzu (kullanıcı)	154
EL	Οδηγίες χρήσης (Χρήστης)	162
RU	Руководство по применению (для пользователей)	170
JA	取扱説明書(使用者)	179
ZH	使用说明书(用户)	187
KO	사용 설명서(사용자)	193

1


1 4G1127=1.9 2 4G1127=12 3 4G1127=5.9 4 4G1127=5.12 5 4G1127 6 4D34

2

3

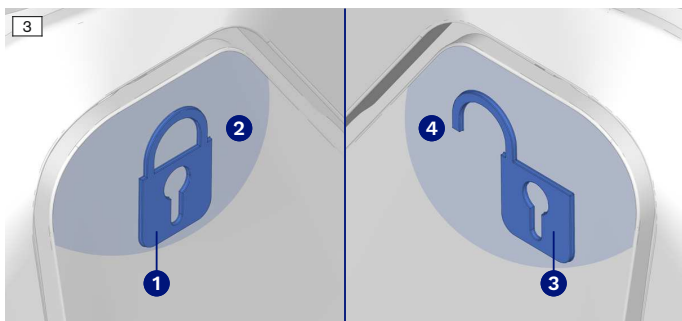5

1

2

3

6

1

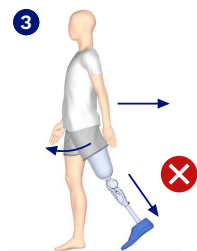
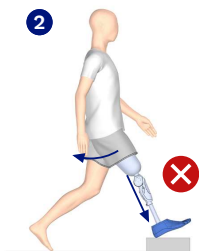
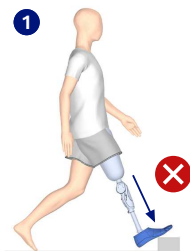
2

3

7

INFORMATION

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INFORMATION

Date of last update: 2026-04-23

- ▶ Please read this document carefully before using the product and observe the safety notices.
- ▶ Obtain instruction from the qualified personnel in the safe use of the product.
- ▶ Please contact the qualified personnel if you have questions about the product or in case of problems.
- ▶ Report each serious incident related to the product to the manufacturer and to the relevant authority in your country. This is particularly important when there is a decline in the health state.
- ▶ Please keep this document for your records.

2 Safety-related information

CAUTION! Risk of injury and risk of product damage

- ▶ To prevent mechanical damage, treat the product with care.
- ▶ Observe the information on product combinations in the instructions for use for the products.
- ▶ Observe the maximum lifetime of the product (see page 16).
- ▶ Observe the product's operating conditions (see page 14).
- ▶ Do not expose the product to prohibited environmental conditions (see page 15).
- ▶ Check the product for damage and readiness for use prior to each use.
- ▶ Do not use the product if it is damaged or if you are not sure whether it is working properly. Have the product inspected, repaired or replaced by the manufacturer or a specialist workshop.
- ▶ Keep the product away from open flames, embers and other sources of heat.
- ▶ The product may become warm during use. Do not touch hot product components.
- ▶ If there are changes in functionality, reduce all activities so the product can cool down.
- ▶ If there are changes in functionality, have the product checked by the manufacturer or a specialist workshop.
- ▶ To avoid the risk of pinching, do not reach into the joint mechanism.
- ▶ In the event of oil leaks, stop using the product and have it repaired.
- ▶ The product is intended for use by one person only and may not be reused by other persons.
- ▶ Follow the maintenance instructions (see page 20).
- ▶ Observe the instructions for cleaning and care (see page 19).

3 Intended use

3.1 Operating conditions

- Permissible body weight: **25 – 60 kg**

The product is recommended for:

- Mobility grade 3: unrestricted outdoor walker
- Mobility grade 4: unrestricted outdoor walker with particularly high demands

3.2 Environmental conditions

Storage and transport

- Storage temperature: -20 °C to +60 °C
- Relative humidity: 20 % to 90 %
- No mechanical vibrations or impacts

INFORMATION

Clean the product after contact with humidity/chemicals/solids, in order to avoid increased wear and tear and damage (see page 19).

Allowable environmental conditions

- Temperature range: -10 °C to +45 °C
- Relative humidity: 20 % to 90 %
- Contact with soap solution (e.g. when taking a shower)
- Contact with perspiration
- Submersion in fresh water/chlorinated water (e.g. swimming pool):
 - Maximum immersion depth: 2 m
- Submersion in salt water with a low salt content (14 days/year, 1 hour/day):
 - Permissible salt content: max. 3.5 % (e.g. ocean)
 - Maximum immersion depth swimming pool: 1 m
 - Maximum immersion depth sea: 0.5 m
- Contact with dust, drifting sand (e.g. walking on the beach)
- Contact with salty, condensing air (e.g. in a 40 °C steam sauna)
- UV resistant

Unallowable environmental conditions

- Contact with liquid-binding particles (e.g. talcum)
- Contact with large amounts of sand and dust (e.g. digging, kneeling in sand, construction site work)
- Contact with acids
- Contact with urine
- Submersion in salt water with excessive salt content
- Salt content: > 3.5 % (e.g. brine bath, Dead Sea)
- Cleaning agents and disinfectants with solvents, chlorine and phosphate
- High water pressure (e.g. diving, jumping into water)

3.3 Lifetime

This prosthetic component has been load tested by the manufacturer according to ISO 10328. The maximum lifetime is 5 years.

4 Accessories

The following components can be additionally ordered from qualified personnel:

3R84=P (see fig. 1)		
Item	Reference number	Designation
①	4G1127=1.9	Front cover (grey)
②	4G1127=12	Front cover (pink)
③	4G1127=5.9	Front cover (light blue)
④	4G1127=5.12	Front cover (azure blue)
⑤	4G1127	Front cover (dark grey)
⑥	4D34	Single component pack for foam cover (push buttons)

5 Use

INFORMATION

If the product is stored at a low temperature (-10 °C to +10 °C), this can lead to changes in functionality due to the cold hydraulic oil. The hydraulic oil heats up again during use of the product.

► Flex and extend the joint several times to warm the oil.

5.1 Front cover

The front cover can be removed for replacement.

> The instructions refer to the following illustration: see fig. 2

1) Open the cover at the lower notch using a blunt object ①.

Alternatively, the cover can be lifted at the side edge using a fingernail ②.

2) Pull the opened cover off the prosthetic knee joint ③.

→ The cover has been removed and can now be replaced.

3) Attach the new cover. Make sure that the studs are properly fitted into the stud openings ④.

→ The cover has been correctly attached.

5.2 Switching between the stance phase and swing phase

- At the end of the stance phase, stance phase flexion resistance ensures a stable, extended knee and prevents unintentional flexion. Once the leg is unloaded and begins to swing forward, the prosthetic knee joint switches to its swing phase flexion resistance.
- Stance phase flexion resistance is activated as soon as the prosthetic knee extends in the swing phase.
- The stance phase flexion resistance prevents unintentional flexion of the prosthetic knee joint.

CAUTION! Fast, hard movements can deactivate stance phase flexion resistance. Therefore, follow the instructions below and avoid the movements mentioned.

- > The instructions refer to the following illustration: see fig. 7
- 1) Avoid a hard forefoot strike, especially with the prosthetic leg extended and subsequent knee flexion (e.g. stepping off a curb) ❶.
 - 2) Avoid fast, forceful steps forward or jumping and do not strongly extend the hip at heel strike followed immediately by knee flexion ❷.
 - 3) Avoid walking backward while loading the prosthetic forefoot and flexing the knee at the same time ❸.

5.3 Using the lock

CAUTION

Locking function

Risk of injury due to unintentional activation or deactivation of the locking function

- ▶ Check that the joint is functioning correctly each time you activate or deactivate the lock.
- ▶ Make sure that the foam cover and clothing cannot activate the lock.

INFORMATION

Noises may occur due to the play between the extension stop and lock pin when walking with the lock activated. The lock can only be released by actively pressing the deactivation button.

Activating the lock

- > Activate the lock for use in wet areas (e.g. showering and swimming) and relaxed standing.
- > The instructions refer to the following illustration: see fig. 3
- 1) Put weight on the prosthetic knee joint and extend it to the stop.
- 2) Press the switch to activate the lock ❶.
 - **TIP:** Press the switch in the marked area ❷; this requires the least amount of operational force.
- The lock is activated.

Deactivating the lock

- > Deactivate the lock for normal walking.
- > The instructions refer to the following illustration: see fig. 3
- 1) Put weight on the prosthetic knee joint and extend it to the stop.
- 2) Press the switch to deactivate the lock ❸.
 - **TIP:** Press the switch in the marked area ❹; this requires the least amount of operational force.
- The lock is deactivated.

5.4 Using cycling mode

CAUTION

Walking in cycling mode

Risk of injury due to lack of stance phase flexion resistance in cycling mode

- ▶ Switch back to standard mode each time after using cycling mode.
- ▶ Carefully verify that standard mode has been activated after switching.

Activating cycling mode

- > Activate cycling mode for cycling and similar movement patterns.
- > The instructions refer to the following illustration: see fig. 4
- ▶ **INFORMATION: The standard mode and the stance phase flexion resistance are deactivated by activating the cycling mode.**
Activate cycling mode by pressing the switch all the way in ❶.
 - **TIP:** Press the switch in the marked area ❷; this requires the least amount of operational force.
 - When the switch for cycling mode is pressed, the switch for normal walking protrudes slightly.
- Cycling mode is activated.

Deactivating cycling mode

- > Deactivate cycling mode for normal walking.
- > The instructions refer to the following illustration: see fig. 4
- ▶ **INFORMATION: Activating standard mode deactivates cycling mode and activates the stance phase extension resistance.**
Deactivate cycling mode by pressing the switch all the way in ③.
 - **TIP:** Press the switch in the marked area ④; this requires the least amount of operational force.
 - When the switch for normal walking is pressed, the switch for cycling mode protrudes slightly.
- The cycling mode is deactivated.

5.5 Use in water

Prior to use in water

NOTICE! Activate the lock and cycling mode before using the product in water to avoid product damage.

- > The instructions refer to the following illustration: see fig. 5
- 1) Activate the lock ①.
- 2) Activate cycling mode as well ②.
- The product can be used in water ③.

After use in water

- > The instructions refer to the following illustration: see fig. 6
- 1) Deactivate cycling mode ①.
- 2) Then deactivate the lock ②.
- The product can be used normally again ③.

5.6 Cleaning

Requirements for cleaning agents and disinfectants
• Solvent-free • Chlorine-free • Phosphate-free

Cleaned using soap and water

- > **Required materials and tools:** 2 microfibre cloths, pH-neutral soap (e.g. 453H10 Derma Clean), tap water
- 1) **In case of light soiling (e.g. small deposits):** Moisten a microfibre cloth with water.
- 2) **In case of heavy soiling (e.g. stubborn deposits):** Moisten a microfibre cloth with pH-neutral soap and water.
- 3) Clean the product with the moistened cloth.

- 4) Dry the product with a dry microfibre cloth.
- 5) Allow the product to air dry.

Cleaning after contact with salt water

NOTICE! Remove salt crystals from the prosthetic knee joint to prevent corrosion.

- > **Required materials and tools:** 2 microfibre cloths, warm tap water
 - 1) Activate cycling mode.
 - 2) Fully immerse the joint in warm tap water and flex and extend it several times.
 - If crystals stick, carefully wipe them off with a microfibre cloth. Immerse the joint in water again and rinse it off.
 - 3) Remove the joint from the water and allow it to drain.
 - 4) Dry it thoroughly with a microfibre cloth.
 - 5) Check the functionality of the joint (flexing, extending, locking).
 - 6) Deactivate cycling mode.
- After maximum use in salt water (**1 hour/day for a maximum of 14 days/year**), have it checked by qualified personnel.

Disinfecting the product

- > **Required materials and tools:** 2 microfibre cloths, water-based disinfectant
 - 1) Apply disinfectant to the microfibre cloth.
 - 2) Disinfect the product with the moistened cloth.
 - 3) Dry the product with a dry microfibre cloth.
 - 4) Allow the product to air dry.

6 Maintenance

- **Every 12 months:** Have maintenance performed by qualified personnel.

7 Disposal

Do not dispose of the product in household waste. Improper disposal is harmful to health and the environment. Observe the instructions of the local authority regarding return, collection and disposal.

8 Technical data

3R84=P	
Proximal connection	Pyramid
Distal connection [mm]	30
Knee flexion angle [°]	145

3R84=P	
Total weight [g]	1217
Overall height [mm]	231
Build height [mm]	216.5
Mobility grade	3, 4
Permissible body weight [kg]	25 — 60

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