
Instructions for Use

ICELOCK® 772 PYRAMID TI
ICELOCK® 773 MALE PYRAMID AL

		3
EN	Instructions for Use	4
DE	Gebrauchsanweisung	10
FR	Notice d'utilisation	17
ES	Instrucciones para el uso	24
IT	Istruzioni per l'uso	31
NO	Bruksanvisning	37
DA	Brugsanvisning	43
SV	Bruksanvisning	49
EL	Οδηγίες Χρήσης	55
FI	Käyttöohjeet	62
NL	Gebruiksaanwijzing	68
PT	Instruções de Utilização	75
PL	Instrukcja użytkowania	81
TR	Kullanım Talimatları	88
RU	Инструкция по использованию	94
JA	取扱説明書	100
ZH	中文说明书	106
KO	사용 설명서	112

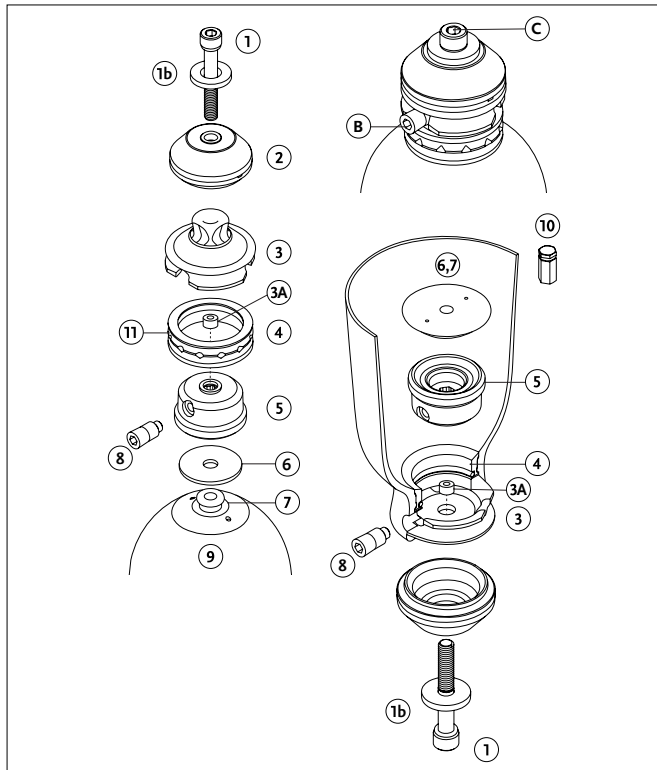ENGLISH

Medical Device

INTENDED USE

The device is intended as a connection component of a prosthetic system that replaces a missing lower limb.

Suitability of the device for the prosthesis and the patient must be evaluated by a healthcare professional.

The device must be fitted and adjusted by a healthcare professional.

For more detailed device information, see **Table 1**.

Indications For Use and Target Patient Population

- Lower limb loss, amputation, or deficiency
- No known contraindications

For weight limit and impact level, see **Table 2**.

GENERAL SAFETY INSTRUCTIONS

Warning: Use of a lower limb prosthetic device carries an inherent risk of falling which may lead to injury.

The healthcare professional should inform the patient about everything in this document that is required for safe use of this device.

Warning: If there is a change or loss in device functionality, or if the device shows signs of damage or wear hindering its normal functions, the patient should stop using the device and contact a healthcare professional.

Warning: Only use Össur replacement parts for the device for ensured durability.

The device is for single patient use.

ASSEMBLY INSTRUCTIONS

Components:

- Pyramid 772/773 (3)
- Plastic Spacer L-772000 only (3A)
- Pyramid Cap (2)
- Pyramid Bolt w/ O-Ring (1)
- M5-M8 Adapter (11)
- Silicone Seal (6)
- Plastic Washer M8 (1b)

Note: When fabrication is complete, insert the following locking component (include Fabrication Ring needed in the fabrication process)

- L-721000 Icelock ratchet 721

Note: Use with L-780000 Fabrication Tooling Kit (reusable)

Lamination Instructions:

(not intended for thermoplastic sockets)

1. Flatten distal end of Plaster Cast (9) to center Steel Guide (7). Attach with double-sided tape or two nails maintaining original pin alignment.
2. Apply inner PVA bag and tie it around “neck” of Steel Guide (7).
3. Trim bag around string and place Silicone Seal (6) over “neck” of Steel Guide (7), onto PVA bag.
4. Screw Fabrication Ring (4) (from locking component kit) onto Lock Dummy (5).
5. Choose appropriate length of Release Dummy (8) and screw it firmly into Lock Dummy (5).
6. Apply release agent to components to enable easy removal after lamination (I-400300 Silicone Adhesive is recommended).

7. Place the components onto Steel Guide (7) (make sure that O-Ring on top of Lock Dummy is in place).
8. Put Pyramid 773 (3) or Pyramid 772 (3) and Plastic Spacer (3A) onto Lock Dummy (5).
9. Place Pyramid Cap (2) and Plastic Washer (1b) onto Pyramid (3) and secure them with Pyramid Bolt (1) (make sure that O-Ring on Bolt is in place).
Note: Be sure to tighten (1-2Nm) so that resin cannot leak in between components.
10. Apply clay into hexagon hole on Release Dummy (B) and Pyramid Bolt (C).
11. Tie lay-up material into grooves on Fabrication Ring (4), Pyramid (3) and Pyramid Cap (2).
Warning: Tie at least two layers of fibers in groove on Pyramid Cap (2).
12. Apply outer PVA bag and pour resin.
13. When curing has finished, remove Bolt (1) and Pyramid Cap (2), grind down to Release Dummy (8) and remove.
Warning: Do not grind material away from distal end of socket since that reduces strength of connection between socket and pyramid.
14. Remove socket from plaster cast.
15. Remove Steel Guide (7) and Silicone Seal (6).
16. Use Hex Driver (10) and socket wrench to release Lock Dummy (5).
17. Press out Plastic Spacer (3A).
18. Insert appropriate locking component according to instructions enclosed.

USAGE

Cleaning and care

Clean with a damp cloth and a mild soap. Dry with a cloth after cleaning.

Environmental Conditions

See which environmental conditions apply to the device in **Table 1**.

A Weatherproof device can be used in a wet or humid environment and can tolerate being splashed by fresh water (e. g., rain), no submersion is allowed.

No contact with salt water or chlorinated water is allowed.

Dry with a cloth after contact with fresh water or humidity.

Clean with fresh water in case of accidental exposure to other liquids, chemicals, sand, dust, or dirt and dry with a cloth.

A Waterproof device can be used in a wet or humid environment and submerged in up to 3-meter-deep water for a maximum of 1 hour.

It can tolerate contact with: Salt water, chlorinated water, perspiration, urine, and mild soaps.

It can also tolerate occasional exposure to sand, dust, and dirt.

Continuous exposure is not allowed.

Dry with a cloth after contact with fresh water or humidity.

Clean with fresh water after exposure to other liquids, chemicals, sand, dust, or dirt and dry with a cloth.

MAINTENANCE

The device and the overall prosthesis should be examined by a healthcare professional. Interval should be determined based on patient activity.

SPECIFICATIONS

Table 1 - Device Specifications

Part #	Description	Material	Environmental Condition
L-773000	Icelock® 773 Male Pyramid AL	Stainless Steel	Weatherproof
L-772000	Icelock® 772 Pyramid Ti	Titanium	Waterproof

REPORT OF SERIOUS INCIDENT

Any serious incident in relation to the device must be reported to the manufacturer and relevant authorities.

DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

LIABILITY

Össur prosthetic devices are designed and verified to be safe and compatible in combination with each other and custom-made prosthetic sockets with Össur adapters, and when used in accordance with their intended use.

Össur does not assume liability for the following:

- Device not maintained as instructed by the instructions for use.
- Device assembled with components from other manufacturers.
- Device used outside of recommended use condition, application, or environment.

Compliance

This device has been tested according to ISO 10328 standard to three million load cycles.

Depending on patient activity, this may correspond to 3-5 years of use.

Table 2 - Usage Specifications

Part #	Impact Level	Weight Limit	ISO 10328 - Label Text
L-773000	The device is intended for children and adolescent patients.	45 kg	ISO 10328 - P3 - 45 kg
L-772000			

ISO 10328 - "P" - "m"kg *)

*) Body mass limit not to be exceeded!

For specific conditions and limitations of use see manufacturer's written instructions on intended use!

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