# allard

Instructions For Use

**Carbon Footplate Rigid**

Art. no. 28036

Instructions for Use / Användarinstruktion  
Brugsanvisning / Anvendelseinstruktion  
Käyttöohjeet / Gebrauchsanweisung  
Gebruiksaanwijzing / Instructions d'utilisation  
Istruzioni per l'uso / Instrucciones de uso  
Kasutusjuhend / Naudojimo instrukcija  
Lietošanas instrukcija / 使用説明書

**Product description**

A carbon footplate that limits the range-of motion (ROM) in the metatarsophalangeal joints (MTP-joints), distributes the pressure in the forefoot and facilitates roll-over.

**Intended use**

The footplate is intended to protect the MTP-joints and the forefoot by limiting the range of motion (ROM), distribute the pressure on the forefoot and facilitate roll-over. The footplate can also be used after immobilizing treatment with a POST-OP shoe or walker in order to gradually increase movement. It can also be used to improve balance after toe amputation. Must be covered by an insole. Single patient, multiple use.

**Indications**

Hallux Rigidus, Hallux Limitus, stress fractures and other injuries or arthritis conditions in the forefoot/midfoot, forefoot trauma, toe sprain and toe amputation.

**Contraindications**

Open ulcers on the foot. Non stable fractures. Weight over 300 lbs/ 135 kg.

**Adjustments**

This product should only be altered by a medical professional.

**Warnings**

Extra precaution is needed if the patient has neuropathy, diabetes or other conditions that may lead to ulcers. Make sure that the carbon footplate is covered by a proper and functional foot orthotic that manage the pressure distribution on plantar surface and that the selected shoe has sufficient inner volume so that pressure sores do not occur.

Fiber reinforced thermoset composite: Do not heat the material

When grinding/cutting: Protect the eyes, use dust protection over nose and mouth, make sure that the edges of the footplate are carefully smoothed and all loose fiber particles are removed. Do not overheat (max. 100° C/212° F) when grinding the composite.

**Material Specifications**

Carbon fibre and fiberglass in an epoxy matrix

**User instruction**

The condition of the foot should be checked daily. In the case of skin irritation or discoloration, discontinue use immediately and contact the fitter/prescriber. If your ability to monitor the foot daily is reduced, ask for assistance. Do not bend the carbon footplate excessively, for example, by squatting.

**Washing Instructions**


Wash with soap and water.

**Disposal**

Disposal in accordance with local legislation.


**Information**

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority in the location where the user and/or patient is established.

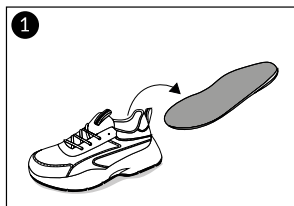**Adjusting the carbon footplate**

Remove the insole from the shoe.


Choose the carbon footplate closest to the patient's shoe size.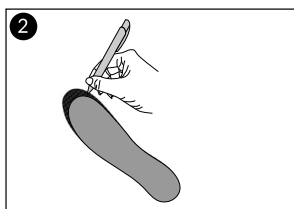


Use the insole from the footwear as a template. Trace the template onto the carbon footplate.

The insole should be a couple of millimeters larger than the carbon footplate.

Grind the carbon footplate with a belt sander or Dremel and smooth the edges.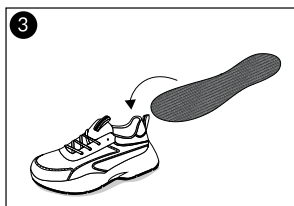

Place the carbon footplate in the shoe.

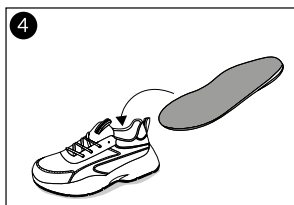Return the shoe insole on top of the carbon footplate.

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©Camp Scandinavia\_0326\_Rev. 03
Camp Scandinavia AB, Karbingatan 38