Instructions for Use

MALE INSERT FOR PRONG JUNIOR
MALE INSERT FOR PRONG
MALE INSERT FOR PRONG TI

EN		Instructions for Use	3
CS		Návod k použití	8
SK		Indikácie použitia	13
RO		Indicații de utilizare	18
HR		Upute za uporabu	23
HU		Használati javallat	28
BG		Указания за употреба	33
SL		Indikácie pre použitie	38

ENGLISH

Medical Device

INTENDED USE

The device is intended as a connection component of a prosthetic system that replaces a missing lower limb.

Suitability of the device for the prosthesis and the patient must be evaluated by a healthcare professional.

The device must be fitted and adjusted by a healthcare professional.

For more detailed device information, see **Table 1**.

Indications For Use and Target Patient Population

- Lower limb loss, amputation, or deficiency
- No known contraindications

For weight limit and impact level, see **Table 2**.

GENERAL SAFETY INSTRUCTIONS

Warning: Use of a lower limb prosthetic device carries an inherent risk of falling which may lead to injury.

The healthcare professional should inform the patient about everything in this document that is required for safe use of this device.

Warning: If there is a change or loss in device functionality, or if the device shows signs of damage or wear hindering its normal functions, the patient should stop using the device and contact a healthcare professional.

The device is for single patient use.

ASSEMBLY INSTRUCTIONS

Warning: For rotation adjustment always engage threads as far as possible whilst maintaining correct rotation of adapter and torque pinch bolt.

USAGE

Cleaning and care

Clean with a damp cloth and a mild soap. Dry with a cloth after cleaning.

Environmental Conditions

See which environmental conditions apply to the device in **Table 1**.

A Weatherproof device can be used in a wet or humid environment and can tolerate being splashed by fresh water (e. g., rain), no submersion is allowed.

No contact with salt water or chlorinated water is allowed.

Dry with a cloth after contact with fresh water or humidity.

Clean with fresh water in case of accidental exposure to other liquids, chemicals, sand, dust, or dirt and dry with a cloth.

A Waterproof device can be used in a wet or humid environment and submerged in up to 3-meter-deep water for a maximum of 1 hour.

It can tolerate contact with: Salt water, chlorinated water, perspiration, urine, and mild soaps.

It can also tolerate occasional exposure to sand, dust, and dirt.

Continuous exposure is not allowed.

Dry with a cloth after contact with fresh water or humidity.

Clean with fresh water after exposure to other liquids, chemicals, sand, dust, or dirt and dry with a cloth.

MAINTENANCE

The device and the overall prosthesis should be examined by a healthcare professional. Interval should be determined based on patient activity.

SPECIFICATIONS

Table 1 - Device Specifications

Part #	Description	Material	Environmental Conditions
AJ-133300	Male Insert For Prong Junior	Aluminum & Stainless Steel	Weatherproof
A-134300	Male Insert For Prong	Stainless Steel	
AJ-135300	Male Insert For Prong Junior	Titanium	Waterproof
A-135300	Male Insert For Prong TI		

REPORT OF SERIOUS INCIDENT

Any serious incident in relation to the device must be reported to the manufacturer and relevant authorities.

DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

LIABILITY

Össur prosthetic devices are designed and verified to be safe and compatible in combination with each other and custom-made prosthetic sockets with Össur adapters, and when used in accordance with their intended use.

Össur does not assume liability for the following:

- Device not maintained as instructed by the instructions for use.
- Device assembled with components from other manufacturers.
- Device used outside of recommended use condition, application, or environment.

Compliance

This device has been tested according to ISO 10328 standard to three million load cycles.

Depending on patient activity, this may correspond to 3-5 years of use.

Table 2 - Usage Specifications

Part #	Impact Level	Weight Limit	ISO 10328 - Label Text
AJ-133300	The device is intended for children and adolescent patients.	45 kg	ISO 10328 - P3 - 45 kg
AJ-135300		60 kg	ISO 10328 - P4 - 60 kg
A-134300	The device is for low to moderate impact use, e.g., walking.	100 kg	ISO 10328 - P5 - 100 kg
A-135300	The device is for low to high impact use, e.g., walking and occasional running.	166 kg	ISO 10328 - P7 - 166 kg

ISO 10328 - "P" - "m"kg ☆)

*) Body mass limit not to be exceeded!

For specific conditions and limitations of use
see manufacturer's written instructions on
intended use!

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