


How to use

Underlay Transtibial (BK)




User Guide

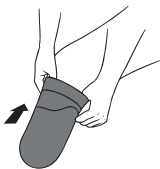

- ① Slide the underlay over your residual limb, Make sur that the underlay is not loose.

The underlay should be fit with no wrinkles.

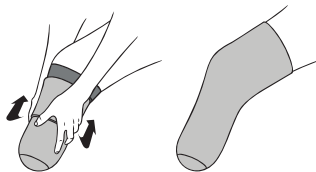

- ② Put on your prosthetic liner over the underlay, ensuring there is enough silicon in contact with your limb over the underlay for optimal suction.

The underlay must be worn completely under the prosthetic liner.

CLOSED

ETHNOCARE™

How it works

Underlay



Moisture wicking

The special knit fabric absorbs humidity from the skin and channels it upward toward the breathable mesh, where it can evaporate more easily.



Stay fresh treatment

Prevents the growth of odor-causing bacteria, keeping your product fresh even in sweat and humidity.



STAYFRESH™
POLYGENE FRESHNESS



STAYCOOL
BY POLYGENE

ETHNOCARE

UNDERLAY

Breathable mesh



The breathable mesh promotes airflow and helps moisture evaporate quickly, keeping you dry and comfortable throughout the day.

Stay cool treatment



Proven to lower fabric temperature by 2-3°C for a refreshing feel.

Washing instructions



Do not use bleach to clean
Wash below 40°C / 104°F
Dry at low temperature
Do not iron.
Do not dry clean.