

ST&G Corporation

www.stngco.com e-mail: info@stngco.com

2691 Saturn Street, Brea, CA 92821, USA
Tel : 1-714-524-0663 Fax : 1-714-364-8113

Room No. 609, 43, Yangsan-ro
Yeongdeungpo-gu
SEOUL 07270
South Korea
Tel : 82-2-6267-8195 Fax : 82-2-6267-8199



Authorized Representative
MDSS GmbH
Schiffgraben 41
30175 Hannover, Germany
Tel: +49 511 6262 8630 Fax: +49 511 6262 8633



REF : P231A

SACH Foot Adapter 22mm, Al
Weight: 28 grams (W/Bolt)
Weight Limit: 55 kg/ 121 lbs
Torque recommendation:
Attachment bolt cap screw: 20 Nm

e-01

<p>Patient Data</p> <p>Height: _____</p> <p>Weight: _____</p> <p>Age: _____</p> <p>Sex: Male <input type="checkbox"/> Female <input type="checkbox"/></p> <p>Length of residual limb: _____ cm</p> <p>Date of fitting: ____ / ____ / ____ .</p> <p>LOT Number: _____</p>	<p>Special remarks regarding the patient</p> <p>_____</p> <p>_____</p> <p>_____</p> <p>_____</p> <p>_____</p> <p>Description of problem/ Reason for return</p> <p>_____</p> <p>_____</p> <p>_____</p> <p>_____</p> <p>_____</p>
---	---

1. Application

The P231A, SACH Foot Adapter 22mm, Al is to be exclusively used for the prosthetic fitting of lower extremity amputations.

2. Description

The SACH Foot Adapter 22mm, Al is used to connect to a foot with single bolt attachment hole. The pyramid is connected to a pyramid receiver connection for anchoring and alignment setting. Apply loctite to the screw once the alignment and connection is finalized.

3. Warranty

Warranty applies only when the product is used as intended without modification, following all manufacturer's recommendations. ST&G offers 1 year warranty.

4. Declaration of Conformity

ST&G as manufacturer declares that the P231A, SACH Foot Adapter 22mm, Al conforms to the requirements of 93/42/ EEC Guideline.

Notice

Tighten the all screws, which are side by side, using the torque wrench