
1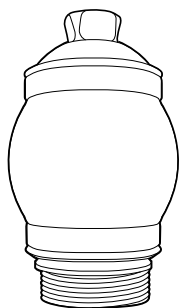

ENGLISH

DESCRIPTION OF SYMBOLS

MD Medical Device

PRODUCT DESCRIPTION

Thank you for choosing an Össur Product
The Torsion Shock Adapter has shock absorption and rotational functions that reduce rotary and shear forces on the residual limb.

INDICATIONS FOR USE

Lower limb amputation.

CONTRAINDICATIONS

None known.

INTENDED USE

This device is a modular component for a lower limb prosthesis. Compatibility of components and the overall prosthesis must be evaluated by a medical professional. This device is for single patient use. The device is for low to high impact use, i.e. all walking speeds and most daily activities.

GENERAL SAFETY INSTRUCTIONS

Inform your patient about everything in this document that is required for safe use of this product.

WARNING: If the device shows signs of damage or wear hindering normal function, the patient should stop using the device and contact a healthcare professional.

WARNING: In cases of functional change or functional loss, the patient should stop using the device and contact a healthcare professional.

PRODUCT SELECTION

Verify that selected variant of the device is suitable for the impact level and weight limit according to the following table.

Weight (kg)	45-77	78-88	89-100
Weight (lb.)	97-170	171-194	195-220
Low impact level	TSA50001	TSA50001	TSA50001
Moderate impact level	TSA50001	TSA50001	TSA50002
High impact level	TSA50001	TSA50002	TSA50003

Weight (kg)	101-116	117-130	131-147
Weight (lb.)	221-256	257-287	288-324
Low impact level	TSA50002	TSA50003	TSA50003
Moderate impact level	TSA50003	TSA50003	N/A
High impact level	TSA50003	N/A	N/A

FITTING

Attach the Torsion Shock Adapter to a threaded M36, Össur adapter.

Refer to the Instructions for use, for the adapter for correct fitting instructions.

USAGE

Clean device with water if it is exposed to sand or similar substances to avoid increased wear or noise.

Dry after cleaning without using heat.

ENVIRONMENTAL CONDITIONS

Weatherproof - Can tolerate light splashing (e.g. rain). Dry thoroughly if device gets wet. No salt water or chlorinated water allowed.

REPORT A SERIOUS INCIDENT

Important notice to users and/or patients established in Europe:

The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

DISPOSAL

All components of the product and packaging should be disposed of in accordance with respective national environmental regulations.

LIABILITY

Össur does not assume liability for the following:

- Device not maintained as instructed by the instructions for use.
- Device assembled with components from other manufacturers.

- Device used outside of recommended use condition, application, or environment.

COMPLIANCE

This device has been tested according to ISO 10328 standard to two million load cycles.

Depending on patient activity this may correspond to 2-3 years of use.

Part no.	Weight (Kg) Moderate impact	Label Text
TSA50001	88	ISO 10328-P4-88 kg
TSA50002	100	ISO 10328-P5-100 kg
TSA50003	130	ISO 10328-P7-130 kg

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