

# Posterior Pace Carbon Fibre AFOs

## Fitting Instructions

1. Remove the inlay and place the Pace inside the shoe.
2. Place the inlay over the footplate. The inlay may be custom, off the shelf, or supplied with the user's footwear, and can be bonded to the footplate if needed.
3. Loosen the laces, slide your foot in, feed the strap through the calf band slot, and fasten using the Trulife tab. If necessary, adjust strap length by removing the tab and trimming with scissors.

To release the U-Pace (FH460) magnetic buckle, pull the toggle forward.

## How to check the fit



Heel Stride



Mid Stance



Toe Off



Perform trial walking as above. Ensure the knee is not hyperextended.



Check footwear is not overly distorted.



Ensure there are no areas of pressure which may lead to lasting sores or ulcerations.



Ensure the patient is comfortable.

## Footplate Fitting Adjustments

For the **M-Pace (FH400)** and **L-Pace (FH450)** models, the footplate can be ground down to fit the patient's shoe. Ensure proper dust extraction and wear appropriate personal protective equipment during this process.

For the **U-Pace (FH460)** model, the footplate can be trimmed with sharp scissors to accommodate various shoe sizes. After trimming, use the supplied sanding sponge to smooth any rough edges.

**U-Pace**

**Trimmable  
Sections**




## Calf Band Fitting Adjustments (FH460 only)

The height of the **U-Pace (FH460)** calf band can be adjusted by releasing the magnetic buckle and removing the pad and the inner screws with the provided hex key.


Set the band to the desired height and align the calf band holes with the composite strut holes.

Tighten the screws securely and replace the pad.

**SM size**

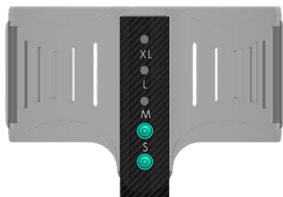

**XL size**

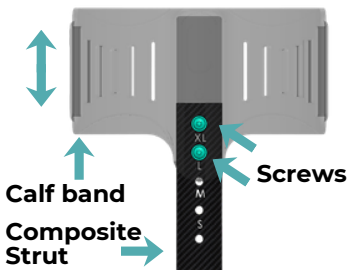

## Intended Use

Pace Products are designed to promote dynamic walking in patients with drop foot or other gait abnormalities.

Pace products should be used only as directed by a medical practitioner.

## Footwear Selection

For optimal performance and durability of the Pace range, footwear should have a firm heel counter, non-slip, flexible sole, and enough room to accommodate the orthosis.

Boots and high heels should not be worn.

Pace products must not be used without footwear.

## Maintenance

The products should be regularly cleaned by wiping with a damp cloth and hand washing the removable pad and strap.



## Important

The product must **NOT** be used for running, squatting, kneeling, climbing ladders, or climbing stairs by loading the footplate unevenly.

Discontinue use immediately if you notice any excessive skin redness, abrasion, or irritation.

Any serious incident should be reported to the manufacturer and the relevant competent authority.

## Disposal

Please dispose of your Pace in accordance with local and national laws and regulations.

## Warranty

**Hardware:** 2 years

**Soft goods:** 90 days

## Any questions? **Contact US**

Whether you have a question about our products, need assistance, or simply want to share your feedback, our team is ready to assist you.

**Scan the QR code  
below for contact  
details.**



Trulife Limited  
41 Amos Road, Sheffield  
S9 1BX, United Kingdom  
[www.trulife.com](http://www.trulife.com)

**EC REP**

MDSS  
Schiffgraben 41, 30175,  
Hannover, Germany

