ENGLISH

DESCRIPTION OF SYMBOLS

MD Medical Device

GENERAL SAFETY INFORMATION

WARNING: In cases of functional change or functional loss, the patient should stop using the device and contact a healthcare professional.

WARNING: If the device shows signs of damage or wear hindering its normal functions, the patient should stop using the device and contact a healthcare professional.

USES

Post surgery, shoulder, wrist, forearm, and elbow injuries, humeral fractures, and to support arm casts.

CAUTION

Patients should be instructed to inspect their shoulder, elbow, hand and this product before and after each use. Modifications to this product are the sole responsibility of the prescribing physician.

PHOTO CAPTIONS

- Shoulder strap
- Sling Pocket
- Contact closures
- D-ring

FITTING INSTRUCTIONS

- Open the sling.
- Place injured arm into the sling. Place elbow deep into the pocket of the sling at a 90° angle when possible. Ensure that the open end of sling supports the hand and fingers.
- Place strap behind the neck and over the opposite shoulder of the injured arm. Position strap and adjust padding on shoulder for comfort.
- Undo contact closure on the end of strap. Loop through d-rings and secure.
- Adjust length of strap using buckle to give proper support. Adjustments may be necessary as swelling patterns change.

WASHING INSTRUCTIONS

- The sling can be hand washed using mild detergent and cold water. Do not wash in washing machine. Do not use bleach.
- Lay sling out flat on a towel and air dry. Do not place in dryer. Avoid direct heat and sun.

REPORT A SERIOUS INCIDENT

Important notice to users and/or patients established in Europe:

The user and/or patient must report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

LIABILITY TEXT

Össur does not assume liability for the following:

- Device not maintained as instructed by the instructions for use.
- Device assembled with components from other manufacturers.
- Device used outside of recommended use condition, application, or environment.

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© Össur 2021. All rights reserved. Össur is a registered trademark of Össur. Össur is not responsible for the content of external websites.