

# AliMed® Protecta-Coat™ Positioners

## Instructions for Use and Care

### Intended Use

The AliMed Protecta-Coat Positioners are intended to maintain the position of an anesthetized patient during surgery. Recommended for use in the operating room (O.R.), imaging, MRI, and radiology.

### Instructions for Use

For use under clinician guidance.

Reusable.

Follow medically recognized patient positioning practices such as those published by the AORN Recommended Practices Committee.

Follow guidelines and standards of practice established by your medical institution.

### Contraindications/Warnings



- Do not use if product is visibly damaged (i.e., the coating is compromised or foam is torn).
- Do not use if patient has known allergies to any of the listed materials.
- Coating color may change slightly with age; this is normal.
- Consult your institution's guidelines for assessing the risk of each individual patient as it pertains to positioning.
- Report any serious incident to AliMed and the appropriate regulatory body.

### Care and Cleaning

Seamless coated positioners are highly resistant to fluid infiltration. Exercise care in their handling and use to maintain the original integrity of the coating. Prevent contact with any sharp objects, items, or edges likely to cut, tear, puncture, pinch, or otherwise compromise the product.

Clean per institution protocol before and after each use. Wipe positioners clean as soon as possible after fluid contact. Hydrogen peroxide ( $\leq 3\%$ ), bleach ( $\leq 10\%$  diluted strength), ammonium chloride and isopropyl alcohol wipes are approved for use with the Protecta-Coat positioners.

- Do not use gas sterilization, autoclaving, washing, or drying machine methods to clean product.
- Do not submerge products in cleaning solution.
- Failure to clean product can lead to infection.
- Using improper cleaning solutions or methods can result in damage to the product or irritation to the patient.
- Use of Povidone-Iodine may cause staining to the surface of the product. This does not affect the performance of the device.



Manufactured for:



**AliMed® Inc.**

297 High St., Dedham, MA 02026 USA  
PH 800.225.2610 • FAX 800.437.2966

AliMed.com

IFU-0065R01 ECO-1459

©2023 AliMed® Inc. All rights reserved

#8097REV0523

## Materials



Finish/Coating: Vinyl with antimicrobial ionic silver  
Foam: Polyurethane (PU) or polyethylene (XLPE)  
Not made with natural rubber latex.

## Storage/Disposal

Store positioners in a flat (natural) position. Store in a clean, dry environment protected from direct sunlight. Storage at ambient room temperature is advised, temperature is not to exceed 170°F. Dispose of in accordance with existing federal, state, and local environmental control laws.

## Symbol Glossary



Caution



Do not use  
if package  
is damaged



Does not contain:  
latex



Non-sterile



MRI Safe



Manufacturer's  
catalog number

## Item Numbers

<b>REF</b>	<b>2970012409</b>	<b>2970012410</b>	<b>2970012411</b>	<b>2970012412</b>	<b>2970012413</b>	<b>2970012414</b>	<b>2970012415</b>	<b>2970012416</b>
(Former Item #)	(924951)	(924952)	(924953)	(924954)	(924955)	(924956)	(924957)	(924958)
<b>REF</b>	<b>2970012417</b>	<b>2970012418</b>	<b>2970012419</b>	<b>2970012420</b>	<b>2970012421</b>	<b>2970012422</b>	<b>2970012424</b>	<b>2970014415</b>
(Former Item #)	(924959)	(924960)	(924961)	(924962)	(924963)	(924964)	(924986)	(936052)
<b>REF</b>	<b>2970014508</b>	<b>2970014548</b>	<b>2970014892</b>	<b>2970014893</b>	<b>2970014894</b>	<b>2970014895</b>	<b>2970014896</b>	<b>2970014897</b>
(Former Item #)	(936679)	(936739)	(937333)	(937334)	(937335)	(937336)	(937337)	(937338)
<b>REF</b>	<b>2970014898</b>	<b>2970015050</b>	<b>2970017827</b>	<b>2970017828</b>				
(Former Item #)	(937339)	(937911)	(960731)	(960732)				

## Warranty

ALIMED warrants to Purchaser that the Product(s) sold hereunder, is at the time of shipment to Purchaser from ALIMED, free from defects in material and workmanship. As Purchaser's sole exclusive remedy under this warranty ALIMED agrees either to repair or replace, at ALIMED's sole option and free of charge to Purchaser, any part or parts of such Product(s) which under proper and normal conditions of use prove to be defective within 6 months from the date of shipment. Visit [AliMed.com/terms](https://www.ali-med.com/terms) for full warranty info.

California Proposition 65 Statement

**65** **WARNING:** This product can expose you to chemicals including Di(2-Ethylhexyl) Phthalate (DEHP), which is known to the State of California to cause cancer and birth defects or other reproductive harm. For more information go to <https://www.p65warnings.ca.gov>.



Manufactured for:



**AliMed® Inc.**

297 High St., Dedham, MA 02026 USA  
PH 800.225.2610 • FAX 800.437.2966

AliMed.com

IFU-0065R01 ECO-1459

©2023 AliMed® Inc. All rights reserved

#8097REV0523