



quick-change adapter
product catalog





Seamless Transitions. Adaptive Freedom.

levitate quick-change adapter

Whether transitioning from daily wear to more active performance, the Levitate Quick-Change Adapter offers a reliable, user-friendly solution that puts versatility at the forefront, so you're always ready for what comes next.

The Levitate Quick-Change Adapter is built to keep up with an active, adaptable lifestyle. With a low build height and universal 4-hole mounting system, it integrates seamlessly into the socket while preserving optimal limb alignment. The switch between prosthetic feet takes just seconds using a standard 4mm Allen key. No realignment or tools beyond that is required.

A precision locking pin mechanism ensures a stable, secure connection, eliminating unwanted movement during activity. Engineered from high-strength, lightweight materials, the adapter supports users up to 150 kg while maintaining durability and comfort.

With the Quick-Change Adapter, switching setups is simple, so you can focus on doing what you love, whenever you want.



Introducing the Levitate Quick-Change Adapter: Designed to make switching prosthetic feet effortless

overview

Have the user go from walking to running in no-time with the **Levitate Quick-Change Adapter**.


The Levitate Quick-Change Adapter is designed to simplify transitions between everyday and high-performance prosthetic setups. Compatible with both the Forever Foot and Running Blades, it allows users to switch components in seconds without modifying the socket. The system consists of four parts - the fastening pin, male connector, female connector, and base plate. Each one can be purchased as a complete kit or as individual components. This streamlined design supports greater mobility and efficiency in everyday life and training.

- **Standard 4-hole System:** The base can be fastened to prosthetic sockets thanks to its four-hole four-hole configuration with a 36 mm spacing.
- **Seamless Compatibility:** Safety and freedom in one solution. Allow users to swap between feet on their own terms with a reliable and secure locking system. Using a 4 mm Allen key, the Q-C adapter allows the user to switch between two compatible foot sizes, such as the Levitate Blade 8" and the Levitate Forever 7". See drawing on page 4 for details
- **Amputation Level:** Transtibial and transfemoral
- **Waterproof Rating:** Water-resistant (not submersible)


maximum user weight

150 kg


component collection

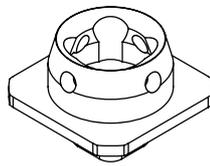


Female QC

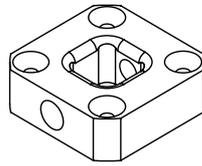

Base QC

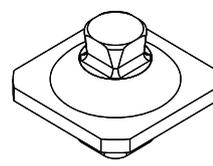

Male QC

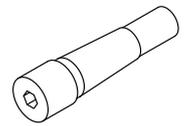

Pin QC

specifications


Component	Material	Weight	Threads
Pin QC	Stainless Steel	25 g	M12
Base QC	Aluminum	85 g	M12
Female QC	Aluminum / Titanium	70 / 118 g	M8
Male QC	Aluminum / Titanium	78 / 124 g	-

user & product info

Max user weight	Warranty	Impact level
150 kg	24 months	Low to High




technical specification (mm)

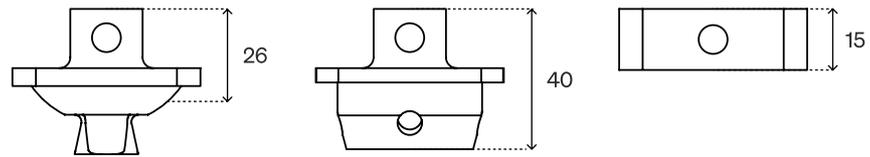


purchase combinations


Male QC + Female QC + Base QC + Pin QC

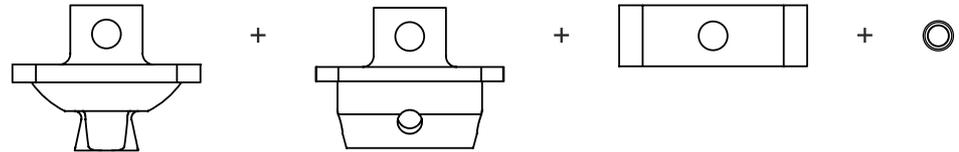

Male QC x 2 + Base QC + Pin QC

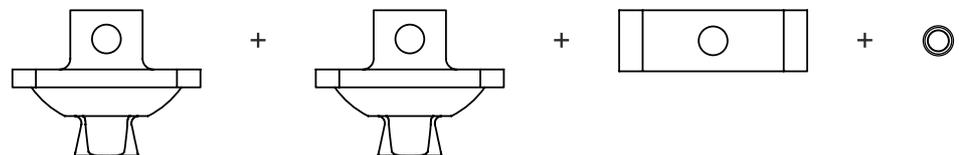


Female QC x 2 + Base QC + Pin QC

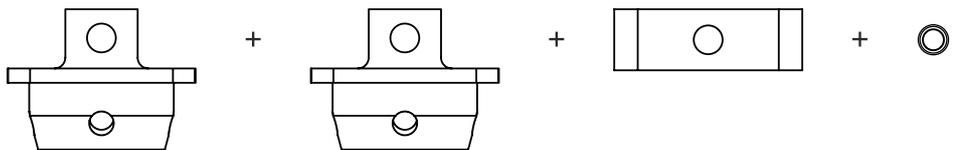

Each component can also be sold separately.
Choose material based on body weight.


compatibility

Blade 8" and Forever 7"

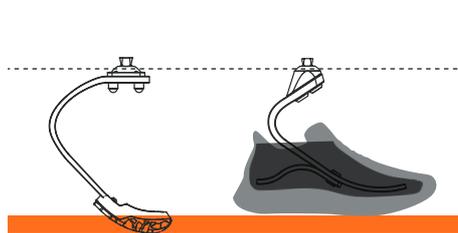

Blade compression zone - 1 in.

Blade 10" and Forever 9"

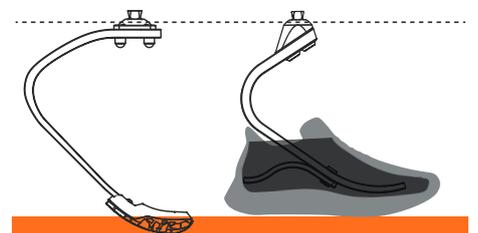

Blade compression zone - 1 in.



material selection

Male QC, Female QC, Base QC Aluminum



Male QC Titanium, Female QC Titanium, Base QC Aluminum



The Levitate Quick-Change Adapter is available in either titanium or aluminum, depending on the user's body weight. For patients up to 125 kg, the lightweight aluminum version is ideal. For those up to 150 kg, we recommend the titanium option for added strength and durability.

product images

Male QC + Female QC + Base QC + Pin QC - Aluminum/Titanium





product images

Female QC x 2 + Base QC + Pin QC - Aluminum/Titanium



Male QC x 2 + Base QC + Pin QC - Aluminum/Titanium





product images



Levitte  Freedom to Play



L5617