

## 4060 - KIT

Post Surgical Bra  
with drain management system

### Product Usage and Care Guidelines



#### Intended Use

- Support for improving comfort and effective drainage management following breast surgery.



#### Important

- Consult your physician regarding the use of this product following surgery and radiation treatment.
- Do not wear this product directly on an open wound.
- Discontinue use if skin irritation occurs.

#### Washing Instructions

- Hand wash in warm water and lay flat to dry.

#### Disposal Instructions

- Dispose in accordance with local and national laws and regulations.



#### Important Notes

Please retain these Instructions for Use for referral. Serious Incidents that occur in relation to these products should be reported to Trulife at [infocanada@trulife.com](mailto:infocanada@trulife.com) and the Competent Authority of your member state.

#### Included:

- 1x 4060 Riley Bra.
- 2x soft triangular seamless puff forms.
- 2x Hook & Loop attachable drain pouches.
- 2x bags of extra fiber filling to customize the fit as needed.



#### Size Chart

4060 Riley Bra	Fiber-filled breast form
34 A/B	01
36 A/B, 34 C/D	02
38 A/B, 36 C/D	03
40 A/B, 38 C/D	04
42 A/B	05
44 A/B	06
40 C/D	07
42 C/D	08
44 C/D	09



Trulife Canada, 39 East Davis Street, Trenton ON K8V 4K8



Trulife Ireland, Airton Road, Tallaght, Dublin 24 R859



Trulife UK, 41 Amos Road, Sheffield, S9 1BX

[www.trulife.com](http://www.trulife.com)

MD



UK  
CA

## 4060 - KIT

Soutien-gorge post-opératoire  
avec système de drainage

### Directives d'utilisation et d'entretien du produit



#### Utilisation prévue

- Soutien pour améliorer le confort et la gestion efficace du drainage après une chirurgie mammaire.



#### Importante

- Consultez votre médecin concernant l'utilisation de ce produit après une intervention chirurgicale ou une radiothérapie.
- Ne pas appliquer ce produit directement sur une plaie ouverte.
- Cesser l'utilisation en cas d'irritation cutanée.

#### Instructions de lavage

- Laver à la main à l'eau tiède et sécher à plat.

#### Consignes pour la mise au rebut

- Veuillez mettre la prothèse mammaire au rebut conformément aux lois et réglementations locales et nationales.



#### Remarques importantes

Veuillez conserver ce mode d'emploi à titre de référence. Les incidents graves qui se produisent en rapport avec ce produit doivent être signalés à Trulife à l'adresse [infocanada@trulife.com](mailto:infocanada@trulife.com) et à l'autorité compétente de votre État membre.

#### Compris:

- 1x soutien-gorge Riley 4060.
- 2x coussinets triangulaires souples sans coutures.
- 2x poches de drainage auto-agrippantes.
- 2x sachets de fibres supplémentaires pour un ajustement personnalisé.



#### Tableau des tailles

Soutien-gorge Riley 4060	Prothèse mammaire remplie de fibres
34 A/B	01
36 A/B, 34 C/D	02
38 A/B, 36 C/D	03
40 A/B, 38 C/D	04
42 A/B	05
44 A/B	06
40 C/D	07
42 C/D	08
44 C/D	09



Trulife Canada, 39 East Davis Street, Trenton ON K8V 4K8



Trulife Ireland, Airton Road, Tallaght, Dublin 24 R859



Trulife UK, 41 Amos Road, Sheffield, S9 1BX

[www.trulife.com](http://www.trulife.com)

MD



UK  
CA