## Rolyan® In-Line® Splint:

A679-001 through A679-008

## Thumb Support:

A679-130 and A679-131

## Rolyan® Hand-Based In-Line® Splint:

A679-111 through A679-118

Figures referred to in the instructions are on page 7.

Be sure to note which splint you are using and follow the instructions that apply to that splint. Unless otherwise stated, these instructions apply to both of the splints.

### INDICATIONS

- To help prevent ulnar drift and subluxation caused by arthritis
- To help prevent further deformity caused by arthritis

### CONTRAINDICATIONS

- Not for people who have an allergic reaction to neoprene or who have a tendency to develop dermatitis
- Not for use over open wounds and infected areas
- May not be for people with circulatory problems

### PRECAUTIONS

- **These splints are to be fitted initially by a healthcare professional who is familiar with the purpose for which they are prescribed. The healthcare professional is responsible for providing wearing instructions and precautions to other healthcare practitioners, care providers involved in the patient's care, and the patient.**
- **If unusual swelling, skin discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.**
- **Be careful that the straps are not so tight that they interfere with circulation.**
- **If advanced deformities are present, they may not be fully correctable and should not be forced.**

### In-Line Splint

- **After heating splinting material, always check its temperature before applying it to the patient.**

### CARE AND CLEANING

#### In-Line Splint

- The stay will lose its shape in temperatures over 135°F (57°C). It should be kept away from sources of heat such as ovens, hot water, open flames, and sunny car windows.

- Remove stay before washing splint.

### In-Line Splint

#### Hand-Based In-Line Splint

- Wash frequently to remove body oils and other soil.
- Fasten hook-and-loop fasteners. Machine or hand wash fabric in warm water and mild soap. Rinse thoroughly. Do not wring. Dry at room temperature. Be sure the splint is completely dry before reapplying it.
- Do not use ointments or oils under the splint.
- Do not dry clean or expose to heat or sunlight for extended periods of time.

### INSTRUCTIONS FOR USE

#### Hand-Based In-Line Splint

##### Applying the splint

1. Place the splint with the hook material facing up.
2. With the volar surface of the hand toward the splint, align the fingers with the four finger web straps and put the thumb through the hole.
3. Wrap the two proximal splint portions over the dorsum of the hand and fasten the hook-and-loop fastener.
4. For finger strap options, see instruction 11.

#### In-Line Splint

##### Modifying optional stay

5. The stay is made from  $\frac{3}{32}$ " (2.4mm) Aquaplast®-T splinting material. Check it for correct fit as shown in **A** and **B** (see page 7). Note the following:
  - It should wrap around the ulnar aspect of the hand, ending just medially and below the ulnar styloid.
  - It should clear the thenar crease.
6. If modification is needed, soften the stay by spot heating it with a heat gun or in a heat pan of water at a temperature of 160° to 170°F (70° to 75°C) for one minute or until transparent.
7. Mold the stay and let it remain on the patient until it is completely cool before putting it into the pocket.

### Applying the splint

8. Place splint onto patient's hand as shown in **C**. Do not insert the hand between the layers of stockinette. Secure the dorsal strap.
9. Be sure the hem portion of the pocket is over the end of the stay.

### In-Line Splint

#### Hand-Based In-Line Splint

##### Fastening the finger straps

10. Wrap the strap around the index finger and fasten it to the splint on the volar side of the hand (**D**).
11. Fasten the remaining finger straps in one of the following manners:
  - Pull each strap across the base of one finger; fasten the straps so each one is stacked on the previous one (**E** and **F**).OR
  - Pull each strap into the web space between two fingers; fasten each one to the splint (**G** and **H**).

### Trimming the splint

12. If trimming the splint, avoid cutting seams and areas with hook or loop material.

### In-Line Splint

#### Adding the thumb piece

13. With the splint in position on the hand, line up the bottom of the U-shaped curve of hook material on the thumb piece with the base of the thumb (**I**). Be sure the hook is not in the web space or directly against the skin.
14. Tuck the edge of the thumb piece that does not have hook into the web space while wrapping the long tab with the hook from the web space to the dorsum of the thumb (**I**).
15. The thumb piece can be repositioned to provide pull in the appropriate direction.

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**Patterson Medical Ltd.**