ST&G Corporation

www.stngco.com e-mail: info@stngco.com

2691 Saturn Street, Brea, CA 92821, USA
Tel : 1-714-524-0663 Fax : 1-714-364-8113

Room No. 609, 43, Yangsan-ro
Yeongdeungpo-gu
SEOUL 07270
South Korea
Tel : 82-2-6267-8195 Fax : 82-2-6267-8199

Authorized Representative
MDSS GmbH
Schiffgraben 41
30175 Hannover, Germany
Tel: +49 511 6262 8630 Fax: +49 511 6262 8633

REF : P211A

English - Pediatric Tube Clamp Assembly 22mm, Al
Spanish - Tubo de extensión con adaptador 22 mm, al.

Weight: 133 g

Weight Limit: 55 kg/ 121 lbs

Torque recommendation:

Alignment adjustment set screw – 6 Nm

8

i-01

1

1. Aplicación

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3. Warranty

Warranty applies only when the product is used as intended without modification, following all manufacturer's recommendations. ST&G offers 1 year warranty.

4. Declaration of Conformity

ST&G as manufacturer declares that the P211A, Pediatric Tube Clamp Assembly, Al conforms to the requirements of 93/42/ EEC Guideline.

Notice

Tighten the all screws, which are side by side, using the torque wrench

6

3

1. Application

The P211A, Pediatric Tube Clamp Assembly 22mm, Al is to be exclusively used for the prosthetic fitting of lower extremity amputations.

2. Description

The Pediatric Tube Clamp Assembly 22mm, Al is used to connect any pyramid system component and 22 mm tube clamp. The tube must be shortened as necessary to fit with patient’s measurement. The cut surface must be perpendicular to the axis of the tube. All burrs from inner and outer surface of the tube must be removed properly. The tube must be fully inserted and completely in contact with the base of the rim inside the tube clamp. The four set screws are used for anchoring the mating pyramid system as well as for making angular adjustment for alignment. Apply recommended torque to pylon clamp screws and alignment adjustment screws for connection. Apply loctite to all screws once the alignment and connection is finalized.

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Date of fitting: _____ / _____ / _____ .

LOT Number: _____

Special remarks regarding the patient

Description of problem/ Reason for return