

ST&G Corporation
 www.stngco.com e-mail: info@stngco.com

2691 Saturn Street, Brea, CA 92821, USA
 Tel : 1-714-524-0663 Fax : 1-714-364-8113

Room No. 609, 43, Yongsan-ro
 Yeongdeungpo-gu
 SEOUL 07270
 South Korea
 Tel : 82-2-6267-8195 Fax : 82-2-6267-8199



Authorized Representative
 MDSS GmbH
 Schiffgraben 41
 30175 Hannover, Germany
 Tel: +49 511 6262 8630 Fax: +49 511 6262 8633



REF : P212A
AL Pediatric Tube Clamp, 22mm

Weight: 31 grams
 Weight Limit: 55 kg/ 121 lbs
 Torque recommendation:
 Alignment adjustment set screw – 6 Nm
 Pylon clamp screw – 4 Nm

e-01

<p>Patient Data</p> <p>Height: _____</p> <p>Weight: _____</p> <p>Age: _____</p> <p>Sex: Male <input type="checkbox"/> Female <input type="checkbox"/></p> <p>Length of residual limb: _____ cm</p> <p>Date of fitting: ____ / ____ / ____ .</p> <p>LOT Number:</p>	<p>Special remarks regarding the patient</p> <p>_____</p> <p>_____</p> <p>_____</p> <p>_____</p> <p>Description of problem/ Reason for return</p> <p>_____</p> <p>_____</p> <p>_____</p> <p>_____</p>
---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------	---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------

1. Application

The P212A, AL Pediatric Tube Clamp, 22mm is to be exclusively used for the prosthetic fitting of lower extremity amputations.

2. Description

The Pediatric Tube Clamp, 22mm is used to connect any component with 22mm tube system. The four set screws are used for anchoring the mating pyramid system as well as for making angular adjustment for alignment. Apply recommended torque to pylon clamp screws and alignment adjustment screws for connection. Apply loctite to all screws once the alignment and connection is finalized.

3. Warranty

Warranty applies only when the product is used as intended without modification, following all manufacturer's recommendations. ST&G offers 1 year warranty.

4. Declaration of Conformity

ST&G as manufacturer declares that the P212A, Pediatric Tube Clamp, 22mm conforms to the requirements of 93/42/ EEC Guideline.

Notice

Tighten the all screws, which are side by side, using the torque wrench