

ST&G Corporation

www.stngco.com e-mail: info@stngco.com

2691 Saturn Street, Brea, CA 92821, USA
Tel : 1-714-524-0663 Fax : 1-714-364-8113

Room No. 609, 43, Yongsan-ro
Yeongdeungpo-gu
SEOUL 07270
South Korea
Tel : 82-2-6267-8195 Fax : 82-2-6267-8199



Authorized Representative
MDSS GmbH
Schiffgraben 41
30175 Hannover, Germany
Tel: +49 511 6262 8630 Fax: +49 511 6262 8633



REF : P220A
4 Ear Connector, Pyramid 22mm, Al

Weight: 17 grams
Weight Limit: 55 kg/ 121 lbs

e-01

<p>Patient Data</p> <p>Height: _____</p> <p>Weight: _____</p> <p>Age: _____</p> <p>Sex: Male <input type="checkbox"/> Female <input type="checkbox"/></p> <p>Length of residual limb: _____ cm</p> <p>Date of fitting: ____ / ____ / ____ .</p> <p>LOT Number: _____</p>	<p>Special remarks regarding the patient</p> <p>_____</p> <p>_____</p> <p>_____</p> <p>_____</p> <p>_____</p> <p>Description of problem/ Reason for return</p> <p>_____</p> <p>_____</p> <p>_____</p> <p>_____</p>
---	--

1. Application

The P220A, 4 Ear Connector, Pyramid 22mm, Al is to be exclusively used for the prosthetic fitting of lower extremity amputations.

2. Description

The 4 Ear Connector, Pyramid 22mm, Al is integrated into the prosthetic socket's laminate as lamination anchor. For reinforcement, tie off the individual layers of stockinette between the lamination dummy and lamination anchor.

3. Reinforcement recommendations

The type of reinforcement used depends on the type of loading expected. It generally consists of four to six layers of Nyglass, some carbon reinforcement and 1-1/2 layers of fiberglass stockinette.

4. Warranty

Warranty applies only when the product is used as intended without modification, following all manufacturer's recommendations. ST&G offers 1 year warranty.

5. Declaration of Conformity

ST&G as manufacturer declares that the P220A, 4 Ear Connector, Pyramid 22mm, Al conforms to the requirements of 93/42/ EEC Guideline.

Notice

Tighten the all screws, which are side by side, using the torque wrench