ENGLISH

MD	Medical Device
-----------	----------------

INTENDED USE

The device is intended to immobilize the shoulder and/or elbow

Indications for use

For shoulder conditions that may benefit from immobilizing the shoulder, such as:

- Pre or post-operative
- Soft tissue repairs/strains
- Shoulder instabilities/injuries

No known contraindications.

GENERAL SAFETY INSTRUCTIONS

Read these instructions carefully before use. Keep them for future reference.

Any serious incident in relation to the device must be reported to the manufacturer and relevant authorities.

Never wear the device directly over an open wound.

The patient should stop using the device and contact a healthcare professional:

- If there is a change or loss in device functionality, or if the device shows signs of damage or wear hindering its normal functions.
- If any pain, skin irritation, or unusual reaction occurs with the use of the device.

The device is for single patient – multiple use.

USAGE

Device Application

- Unfasten the hook straps of the shoulder straps.
- Place forearm into the device and position against the rib cage. Bring the shoulder straps straight up and over the shoulders (**Fig. 1**).
- Place the shoulder straps criss cross against the back (**Fig. 2**) and fasten the hook straps onto the front of the device (**Fig. 3**).
- Adjust the strap length to adjust the elbow to a 90° position.

- Secure the arm against the body by attaching the swathe strap to the device. Place one end of the strap to the device. Encircle the affected arm and bring it around the back. Secure loose end to the front of the sling (**Fig. 4**).

Device Removal

Unfasten the Swathe strap first. Unfasten the hook straps of the shoulder straps to remove the arm out of the device.

Cleaning and care

- Machine-wash with a mild detergent at cold temperature (30°C).
- Air dry.

Note: Secure the hook and loop fasteners before washing to avoid damaging the textiles.

Note: Do not tumble dry, iron, bleach, or wash with fabric softener.

Note: Avoid contact with salt water or chlorinated water. In case of contact, rinse with fresh water and air dry.

DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

LIABILITY

Össur does not assume liability for the following:

- Device not maintained as instructed by the instructions for use.
- Device assembled with components from other manufacturers.
- Device used outside of recommended use condition, application, or environment.

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Össur UK Ltd