



SOCKET ADAPTER 4I42

SOCKET ADAPTER

SOCKET ADAPTER

stainless steel

3 Arm fixed socket adapter. They are used exclusively for the manufacture of lower extremity prostheses.

Made of high resistance stainless steel. Indicated for patients with a maximum weight of 100 kg.

Fixed pyramidal receiving ring adapter with twist fit, used in combination with any pyramid receiving adapter, for modular leg prostheses.

These combinations provide angle and translation modifications in the sagittal and anterior plane, as well as regulation of internal and external rotation.



ACTIVITY LEVEL				
PATIENT WEIGHT LIMIT				
IMPACT LEVEL	1-4	RESISTANCE	100 KG	
SKU	4I42	UNIT WEIGHT	137 GR	

CAUTION!

Do not expose the prosthetic components to environments that favor the corrosion of metal, as well as the degradation of the elements that make up the adapter. If it is exposed to any of these environments and the adapter has traces of elements that favors its deterioration, the warranty offered by O and P will be lost.

WARRANTY

The 4I42 adapter warranty only applies if it is used under the conditions described in this sheet. This product is insured for 1 year using exclusively O and P modular products.

SOCKET ADAPTER

INSTRUCTIONS FOR USE

3 Arm fixed socket adapter with a pyramid receiver.



The union with the opposite piece in the modular system is through the M8 x 14 mm opressors.

Corrections can be made at any time during alignment, try-in and after completion of the prosthesis. To exchange any part, disassemble it. To keep the previous alignment, unscrew only the two adjoining opressors.







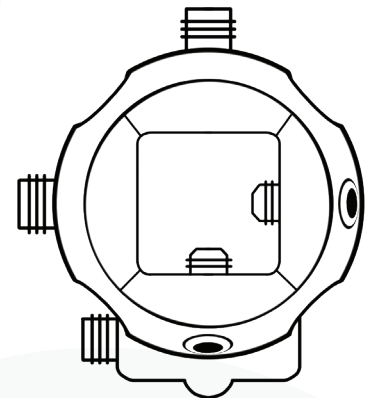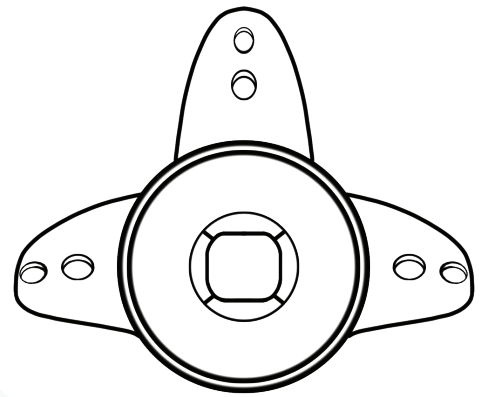

For the definitive use, secure it with a torque wrench set to 15 NM. The application of Loctite 242 threadlocker is necessary.

CAUTION!

Do not clamp the adapter in the workbench screw, as this action can cause deformations or damage.



1

ACTIVITY LEVEL	   
PATIENT WEIGHT LIMIT	   
IMPACT LEVEL 1-4	RESISTANCE 100 KG
SKU 4I42	UNIT WEIGHT 137 GR