

# Fitting & Care Instructions

**Model: DCT-200**

Instructions on putting on brace:



Step 1: Unstrap brace and slide injured hand into sleeve with the palm down.



Step 2: Move large Velcro platform for tighter fit if needed. Tighten and adhere top and bottom strap to desired support



Step 3: Wrap inner thumb strap around thumb and adhere to opposite side.



Step 4: Wrap outer thumb strap and adhere to opposite side



Step 5: Tighten thumb strap around thumb at desired support

## CARE INSTRUCTIONS

**Hand wash with mild detergent and air dry.**

## WARRANTY

**The DCT-200 is covered by a 6 month warranty against all mechanical or manufacturer defects, from the date of purchase.**

**TREND™**  
M E D I C A L