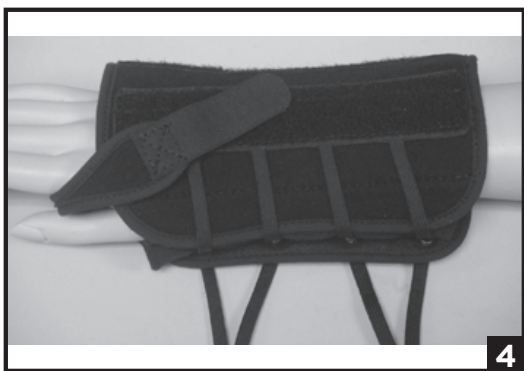
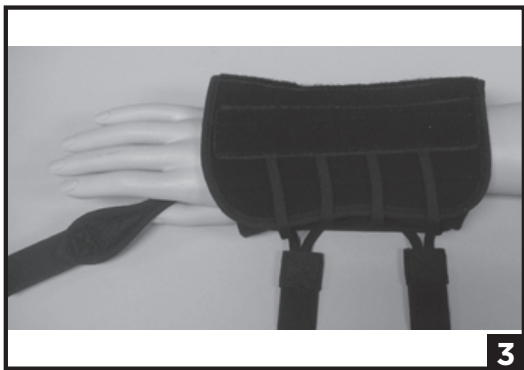
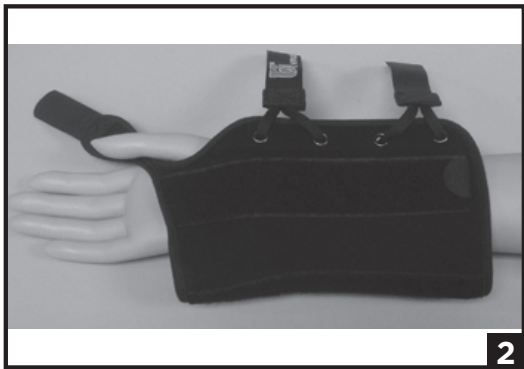
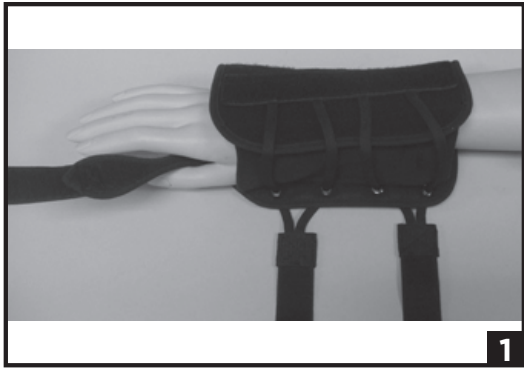
ENGLISH

INTENDED USE/INDICATIONS:

The intended use wrist bracing products is to provide stability, support, and / or immobilization to the rist joint for patients recovering from a wrist injury or surgery.

INTENDED USERS:

The intended user should be a licensed professional, the patient, or patient's caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

CONTRAINDICATIONS:

Not Applicable.

⚠ WARNINGS AND PRECAUTIONS:

- Activity modification and proper rehabilitation are an essential part of a safe treatment program. If any additional pain or other symptoms occur while using this device, discontinue use and seek medical attention.
- This device is not intended to prevent injury, or reduce or eliminate the risk of re-injury.
- Single Patient use.

INSTRUCTIONS FOR USE:

APOLLO WRIST BRACE UNIVERSAL

- Open brace and position hand in wrist brace.
- The palmar stay should line up below the middle crease of the hand.
- Align the lacing dorsal stay with the middle of the knuckles.
- Secure the thumb strap to the brace, aligning it between the thumb and index finger.
- Wrap the lacing straps around the wrist and attach to hook and loop. Re-adjust if necessary. The knuckles should be free for bending.

APOLLO WRIST BRACE WITH SPICA®

Steps 1-3 follow instructions above.

- Align the lacing dorsal stay with the middle of the knuckles.
- Wrap the wide thumb strap around the thumb and secure to hook.
- Secure the thinner strap between the thumb and index finger and attach to hook.
- Wrap the lacing straps around the wrist and attach hook and loop. Re-adjust if necessary. The knuckles should be free for bending.

USE AND CARE:

Hand wash in 30°C (86°F) water with mild soap, rinse thoroughly. Rinse and lay flat to air dry. Do not machine wash or tumble dry.

MATERIAL CONTENT:

Aluminum, Cotton, Nylon, Polyester.

WARRANTY:

Breg products come with a limited warranty against manufacturing and material defects. No other warranty, expressed or implied, is given and no affirmation of or by seller, by words or action, will constitute a warranty. Please see www.breg.com

DISPOSAL:

Dispose of product according to local regulations.

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