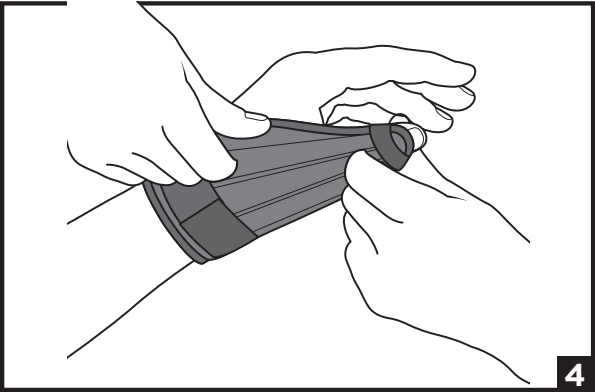
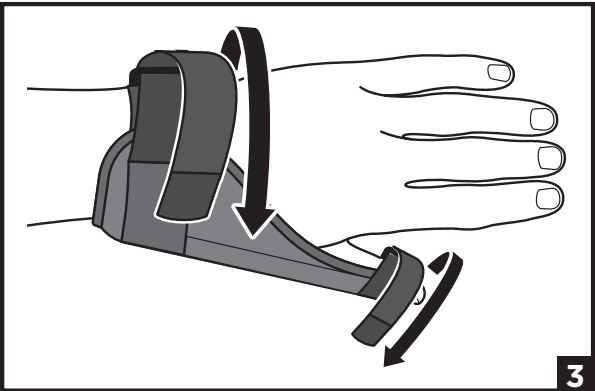
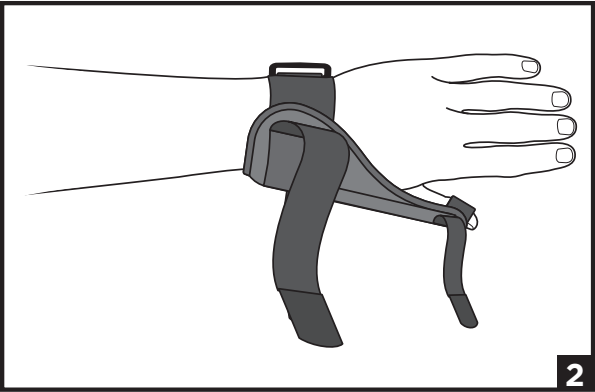
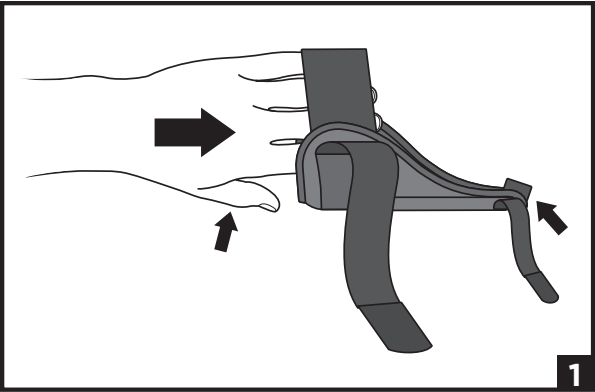
ENGLISH

INTENDED USE/INDICATIONS

The intended use for thumb bracing products is to provide support and / or immobilization to the thumb for patients with thumb injury.

INTENDED USERS:

The intended user should be a licensed professional, the patient, or patient’s caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

CONTRAINDICATIONS:

Not Applicable.

⚠ WARNINGS AND PRECAUTIONS:

- Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.
- If you experience, or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.
- This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.
- For single patient use only.

INSTRUCTIONS FOR USE:

Example illustrates left hand application. Splint can be worn on either hand.

1. Slide affected hand through wrist band and into thumb orthosis. Be sure thumb and thumb strap at the front of the orthosis are aligned.
2. Illustration shows hand and thumb properly positioned in the orthosis.
3. Thread wrist strap through D-ring and pull the strap snug but not too tight. Attach strap end to thumb orthosis. Then wrap the thumb strap around the thumb as shown.
4. For improved fit and comfort, the metal stays can be molded around the thumb by bending the malleable material.

USE AND CARE:

Fasten hook and loop straps. Hand wash in 30°C (86°F) water with mild soap, rinse thoroughly. Rinse and lay flat to air dry. Do not machine wash or tumble dry.

MATERIAL CONTENT:

Acetal, Aluminum, Cotton, Nylon, Polyester, Polyurethane.

WARRANTY:

Breg products come with a limited warranty against manufacturing and material defects. No other warranty, expressed or implied, is given and no affirmation of or by seller, by words or action, will constitute a warranty. Please see www.breg.com

DISPOSAL:

Dispose of product according to local regulations.

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