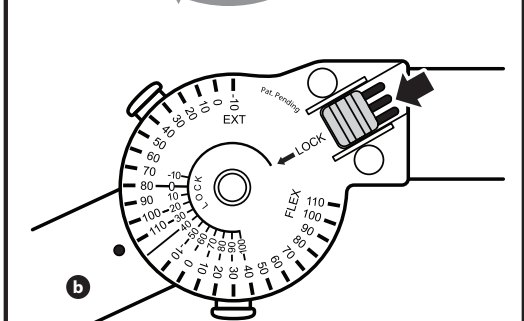
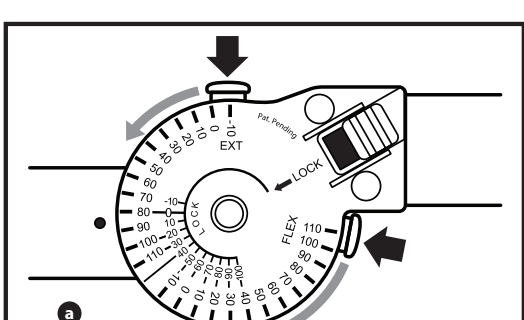
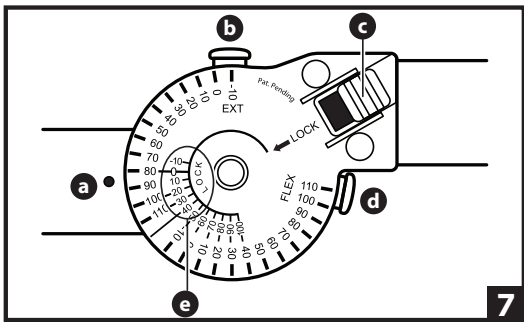
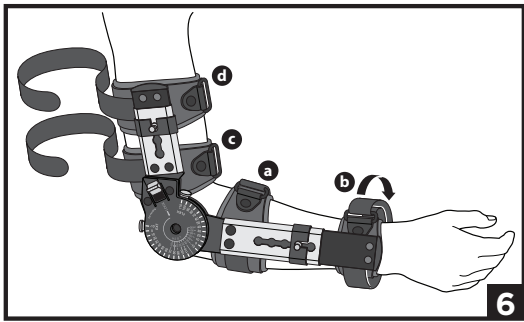
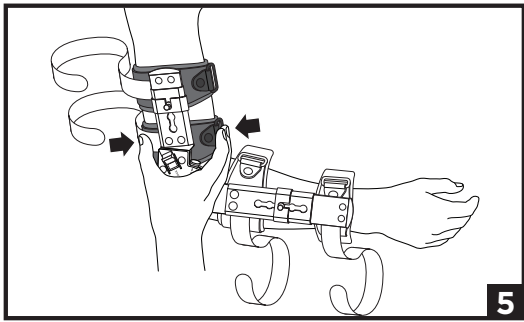
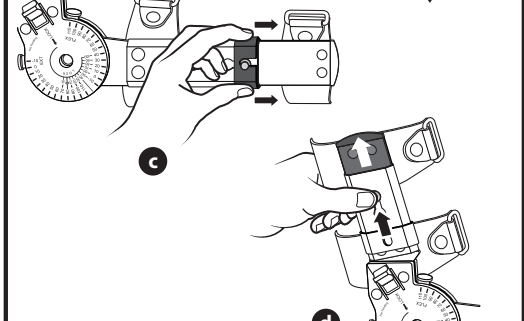
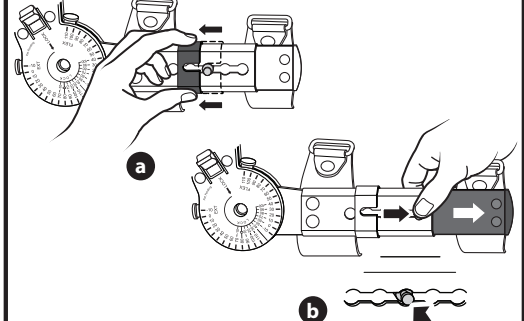
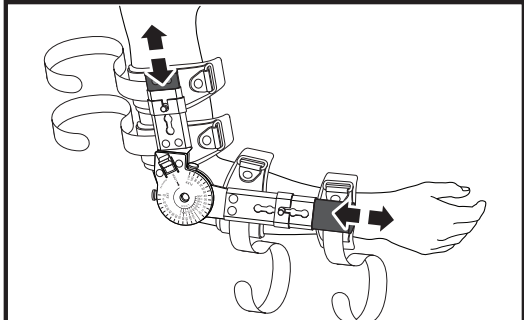
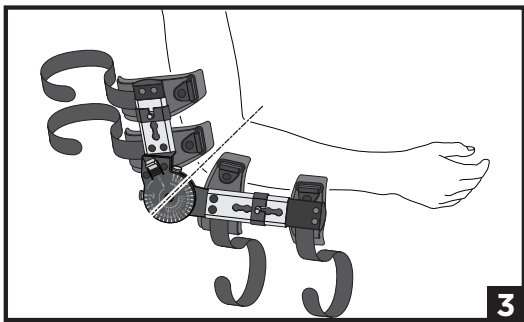
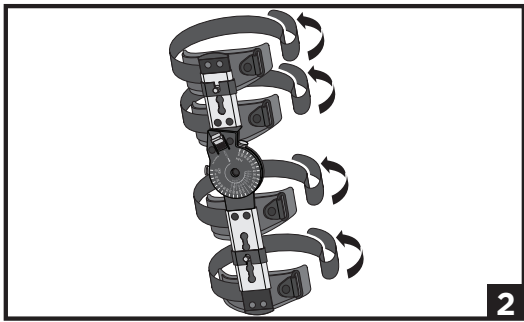
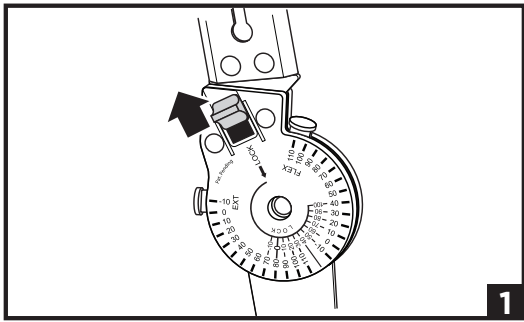
ENGLISH

INTENDED USE/INDICATIONS:

Provides fixed or range of motion control following injury or post-operative procedure to the elbow; stable fractures; biceps repair; and mild to severe soft tissue injuries.

INTENDED USERS:

The intended user should be a licensed professional, the patient, or patient's caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

CONTRAINDICATIONS:

Contraindicated for unstable fractures or for fractures of the proximal humerus, distal radius or ulna.

WARNINGS AND PRECAUTIONS:

- This device is offered for sale by or on the order of a physician or other qualified medical professional. This device is not intended for reuse on a second patient or for extended use beyond 180 days. This device is not intended to prevent injuries or reduce or eliminate the risk of injury.
- Do not leave this device in a car on a hot day or in any other place where the temperature may exceed 150°F (65°C) for any extended period of time.
- If any additional pain or symptoms occur while using this device, discontinue use and seek medical attention.

INSTRUCTIONS FOR USE:

Note: It is easier to apply the brace if it is locked at 90 degrees of flexion. The strap is designed with one end of the strap connected to the malleable aluminum cuff. The other end of the strap has a hook tab that will double back on itself after it is inserted into the D-ring. The strap and that is attached to the cuff can be disengaged and trimmed if a shorter strap is desired.

1. UNLOCK THE HINGE

The WeeBow Elbow Brace is packaged in the locked position. Slide the Lock button away from the hinge to unlock the brace.

2. RELEASE ALL FOUR STRAPS

Pull each strap through the D-ring and attach it back on itself to prevent entanglement.

3. POSITION THE BRACE

Position the hinge over the lateral humeral epicondyle (outer elbow).

4. SET THE ARM LENGTH

- While the hinge is aligned with the elbow (see Step 3), make length adjustments to the forearm.
- a. Slide the locking clip over to expose the length adjustment button.
- b. Depress the length adjustment button and extend the plate to the desired length (forearm).
- c. Secure the locking clip by sliding it over the length adjustment button to lock the button in place.
- d. Repeat the same process to adjust the humeral arm length.

5. APPLY THE BRACE

Position the brace, be sure that the hinge is centered over the elbow joint. Form the black aluminum plates around the forearm and upper arm.

6. SECURE THE STRAPS

Pull the strap closest to the elbow on the forearm (a) over the medial side of the arm, feed the tab end of the strap through the D-ring and pull to the desired tension. This will create a ring around the arm. This secures the position of the brace so it does not move from the desired position while the remainder of the straps are secured. If the strap is too long refer to the above note. Repeat the process with the strap closest to the wrist (b). Repeat the process with the strap above the elbow (c). Repeat the process with the strap closest to the shoulder (d).

7. HINGE INSTRUCTIONS

- Make sure to check the physician's ROM orders and adjust the hinge to those settings.
- a. Indicator Mark
- b. Extension Limit Button (ROM)
- c. Extension Lock Tab
- d. Flexion Limit Button (ROM)
- e. Locking Degrees of Extension

8. SET RANGE-OF-MOTION (ROM)

- a. Depress Extension Button and slide to desired ROM setting. Depress Flexion Button and slide to desired ROM setting. NOTE: ROM settings automatically lock into place.
- b. LOCK HINGE

The Lock Hinge allows the brace to be locked at a specific setting. The hinge can be locked in 10° increments ranging from -10 to 100°. Bend the arm to desired setting and engage lock. This will lock the brace in place independent of ROM settings. Make sure ROM settings do not interfere with desired lock setting.

USE AND CARE:

Hand wash in 30°C (86°F) water with mild soap, rinse thoroughly. Rinse and lay flat to air dry. Do not machine wash or tumble dry.

MATERIAL CONTENT:

Acrylonitrile Butadiene Styrene (ABS), Aluminum, Nylon, Polycarbonate (PC), Polyester, Polytetrafluoroethylene (PTFE), Polyurethane, Steel

WARRANTY:

Breg products come with a limited warranty against manufacturing and material defects. No other warranty, expressed or implied, is given and no affirmation of or by seller, by words or action, will constitute a warranty. Please see www.breg.com

DISPOSAL:

Dispose of product according to local regulations.

NOTICE TO USERS: If any serious incident has occurred in relation to the device, report to Breg and the competent authority of the Member State in which the user and/or patient is established.

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