Explanation of Symbols

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Explication des symboles

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Verklaring van de symbolen

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SEMBOLLERIN AÇIKLAMASI

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PRUVENTOR™/ PRUVENTOR™ XL HEEL OFF-LOADING DEVICE

**RX ONLY
 SINGLE PATIENT USE
 NON-STERILE
 NOT MADE WITH NATURAL RUBBER LATEX**

IMPORTANT INFORMATION

Please read all instructions, warnings, and precautions before use. Correct application is essential for proper functioning of the product.

Use only on the person it was provided to by a healthcare professional and only for the use it was intended.

INTENDED USE

The DeRoyal PRUventor™ or PRUventor™ XL Heel Off-loading Device is intended to be prescribed by a properly licensed practitioner to help prevent heel decubitus ulcers, foot drop and foot rotation.

CONTRAINDICATIONS

The PRUventor™ or PRUventor™ XL Heel Off-loading Device is for non-ambulatory patients only and should be removed before ambulation. Do not allow your patients to stand or walk while wearing the boot.

WARNINGS

- Inspect device for damaged or missing components prior to patient application.
- Consult physician or properly licensed practitioner immediately if you experience sensation changes, unusual reactions, swelling or increased pain while using this product.
- Discontinue use and consult physician or properly licensed practitioner if device does not off-load the heel, or help prevent foot drop or foot rotation (if used with a wedge.)
- If device is used with a sequential/intermittent compression device, be sure tubing does not interfere with off-loading of the patient's foot.

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ankle or calf.

- Discontinue use of the device and consult physician or properly licensed practitioner if it becomes damaged in any way, limiting its ability to perform as intended.

CAUTIONS

- This product is to be fitted initially by a physician (or properly licensed practitioner) who is familiar with the purpose for which it is intended. The physician or practitioner is responsible for providing wearing instructions and precautions to other healthcare practitioners or healthcare providers involved in the patient's care, as well as the patient.
- Follow the instructions of your physician (or properly licensed practitioner) for length and duration of use.
- Be sure to follow your facility's policies and guidelines for frequency of patient monitoring.

INSTRUCTIONS FOR USE

- Open all strap closures and secure the hook tabs to the loop on either side of the boot. The DeRoyal sew-in label should be on the outside of the boot.
- Gently place the patient's foot into the boot, making sure the heel is placed over the heel opening.
- Take one strap and laterally attach to the same side of the boot. Secure hook and loop on the same side of the boot.
- Take the other strap and laterally attach to the same side of the boot. Secure hook and loop on the same side of the boot.
- Take the lower leg straps and apply the hook tab(s) to the loop on either side of the boot. Caregiver can choose to use one or both straps to assist in holding boot on patient's foot by suggested strapping configuration shown above.
- Make sure two fingers fit between the device and the patient to ensure circulation is not compromised. Pull flaps on either side of the foot, toward the patient to ensure the product fits securely and that the heel is suspended over the

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heel opening.

- If the PRUventor™ or PRUventor™ XL Heel Off-loading Device is used with sequential or intermittent compression devices, make sure the tubing exits through the top of the boot or through the access ports located on either side of the boot.
 - Make sure the heel is suspended. Check the bottom of the boot to ensure that nothing is touching the heel.
 - If wedge is included: To apply the anti-rotation wedge, place the hook side of the anti-rotation wedge on the outside, making sure that the bottom of the wedge sits flat on the surface of the bed.
- NOTE:** Perform periodic skin assessment as specified by hospital policies and procedures.

Application videos are available on www.deroyal.com.

SIZING

Universal Short	9" (23cm) Long
Universal Long	13" (33cm) Long
Universal XL	15" (38cm) Long

CALF CIRCUMFERENCE:
 Short & Long Up to 18"
 XL 18" and up

CLEANING AND/OR MAINTENANCE

Wipe down surface using a cloth with soap and water or hospital wipe approved for use on skin. Do not use bleach or oxidizing agents.

WARRANTY

DeRoyal products are warranted for one hundred twenty (120) days from the date of shipment from DeRoyal as to product quality and workmanship. **DEROYAL'S WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

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Part #0-1856G
 Revised 6/2016
 ©2016 PRUventor, DeRoyal Industries, Inc. All Rights Reserved. DeRoyal, the DeRoyal logo, and Improving Care. Improving Business. are trademarks or registered trademarks of DeRoyal Industries, Inc.

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