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KNEE SLEEVE

APPLICATION INSTRUCTIONS

Before applying the brace, please read these instructions completely and carefully. Correct application is vital to the proper functioning of the brace.

INDICATIONS: For general compression and support.

- A** Grasp edges of the sleeve at the larger opening. Bring the sleeve onto the knee so that the knee cap is centered within the brace. For padded version, pad should cover the front of the knee.

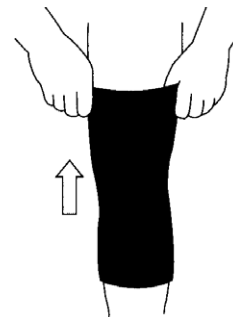
Figure 1

Figure 2

USE AND CARE

- Hand wash in cold water with mild soap, rinse thoroughly. AIR DRY. Note: If not rinsed thoroughly, residual soap may cause skin irritation and deterioration of the material.

WARNING: NEOPRENE SUPPORTS SHOULD NOT BE WORN BY INDIVIDUALS WITH KNOWN SUSCEPTIBILITY TO DERMATITIS OR ALLERGIES TO NEOPRENE BY-PRODUCTS.

WARNING: IF YOU EXPERIENCE ANY PAIN, SWELLING, SENSATION CHANGES, OR ANY UNUSUAL REACTIONS WHILE USING THIS PRODUCT, CONSULT YOUR MEDICAL PROFESSIONAL IMMEDIATELY.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT. THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

dj Ortho and DonJoy are trademarks of dj Orthopedics, LLC
dj Ortho y DonJoy es una marca registrada de dj Orthopedics, LLC
dj Ortho et DonJoy est une marque déposée de dj Orthopedics, LLC
dj Ortho und DonJoy ist ein eingetragenes Warenzeichen von dj Orthopedics, LLC
dj Ortho e DonJoy sono marchi depositati dalla dj Orthopedics, LLC

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© dj Orthopedics, LLC