Instructions for Use

Male Single Adapter AL

Male Single Adapter TI

Female Single Adapter AL

Female Single Adapter TI

EN		Instructions for Use	3
DE		Gebrauchsanweisung	9
FR		Notice d'utilisation	16
ES		Instrucciones para el uso	22
IT		Istruzioni per l'uso	28
NO		Bruksanvisning	34
DA		Brugsanvisning	40
SV		Bruksanvisning	46
EL		Οδηγίες Χρήσης	52
FI		Käyttöohjeet	59
NL		Gebruiksaanwijzing	65
PT		Instruções de Utilização	71
PL		Instrukcja użytkowania	77
TR		Kullanım Talimatları	83
RU		Инструкция по использованию	89
JA		取扱説明書	95
ZH		中文说明书	101
KO		사용 설명서	107

ENGLISH

Medical Device

INTENDED USE

The device is intended as a connection component of a prosthetic system that replaces a missing lower limb.

Suitability of the device for the prosthesis and the patient must be evaluated by a healthcare professional.

The device must be fitted and adjusted by a healthcare professional.

For more detailed device information, see **Table 1**.

Indications For Use and Target Patient Population

- Lower limb amputation and/or congenital deficiency
- No known contraindications

For weight limit and impact level, see **Table 2**.

GENERAL SAFETY INSTRUCTIONS

The healthcare professional should inform the patient about everything in this document that is required for safe use of this device.

Warning: If there is a change or loss in device functionality, or if the device shows signs of damage or wear hindering its normal functions, the patient should stop using the device and contact a healthcare professional.

Warning: Only use Össur replacement parts for the device for ensured durability

The device is for single patient use.

ASSEMBLY INSTRUCTIONS

Warning: Ensure tube is fully inserted into Tube Clamp Adapter before tightening.

Only applicable to the Male Single Adapters Short (A-433110, A-435110):

Warning: Do not cut adapter.

Applicable to other devices in **Table 1**:

Cutting the tube:

1. Clamp the tube with a cutting jig or put into a tube cutter.

Caution: Clamping too hard may damage the device.

2. Cut the tube straight and perpendicular.

Warning: Do not cut adapter shorter than the minimum cutting length, see **Table 1**.

3. Remove all burrs and fully insert the tube into adapter.

Additional step for **Female Single Adapters**:

4. **Warning:** For secure fastening use medium strength threadlocker and apply rated torque twice, see **Table 1**.
5. **Warning:** When extreme alignment is necessary, use provided extended set screws to achieve secure fastening.

USAGE

Cleaning and care

Clean with a damp cloth and a mild soap. Dry with a cloth after cleaning.

ENVIRONMENTAL CONDITIONS

See which environmental conditions apply to the device in **Table 1**.

A Weatherproof device can be used in a wet or humid environment and can tolerate being splashed by fresh water (e.g., rain), no submersion is allowed.

No contact with salt water or chlorinated water is allowed.

Dry with a cloth after contact with fresh water or humidity.

Clean with fresh water in case of accidental exposure to other liquids, chemicals, sand, dust, or dirt and dry with a cloth.

A Waterproof device can be used in a wet or humid environment and submerged in up to 3-meter-deep water for a maximum of 1 hour.

It can tolerate contact with: Salt water, chlorinated water, perspiration, urine, and mild soaps.

It can also tolerate occasional exposure to sand, dust, and dirt.

Continuous exposure is not allowed.

Dry with a cloth after contact with fresh water or humidity.

Clean with fresh water and dry with a cloth after exposure to other liquids and chemicals or in case of accidental exposure to sand, dust, or dirt.

MAINTENANCE

The device and the overall prosthesis should be examined by a healthcare professional. Interval should be determined based on patient activity.

SPECIFICATIONS

Table 1 - Device Specifications

Part #	Description	Material	Tightening Torque	Environmental Condition	Minimum Cut Length
			Set Screws		
A-433110	Male Single Adapter Short	Aluminum, Stainless Steel	None	Weatherproof	Do not cut
A-433120	Male Single Adapter Long		None		70 mm
A-442100	Female Single Adapter Short	Aluminum	15 Nm		60 mm
A-442110	Female Single Adapter Long		15 Nm		60 mm
A-442420	Female Single Adapter 5mm offset		15 Nm		64 mm
A-435110	Male Single Adapter Short	Titanium	None		Waterproof
A-435120	Male Single Adapter Long		None	70 mm	
A-445100	Female Single Adapter		15 Nm	60 mm	

REPORT OF SERIOUS INCIDENT

Any serious incident in relation to the device must be reported to the manufacturer and relevant authorities.

DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

LIABILITY

Össur does not assume liability for the following:

- Device not maintained as instructed by the instructions for use.
- Device assembled with components from other manufacturers.
- Device used outside of recommended use condition, application, or environment.

Compliance

This device has been tested according to ISO 10328 standard to three million load cycles.

Depending on patient activity, this may correspond to 3-5 years of use.

Table 2 – Usage Specifications

Part #	Impact Level	Weight Limit	ISO 10328 - Label Text
A-433110	The device is for low to moderate impact use, e.g., walking.	100 kg	ISO 10328 – P5 – 100 kg
A-433120			
A-442100			
A-442110			
A-442420			
A-435110	The device is for low to high impact use, e.g., walking and occasional running.	166 kg	ISO 10328 – P7 – 166 kg
A-435120			
A-445100			

ISO 10328 - “P” - “m”kg *)

*) Body mass limit not to be exceeded!

For specific conditions and limitations of use see manufacturer's written instructions on intended use!

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