


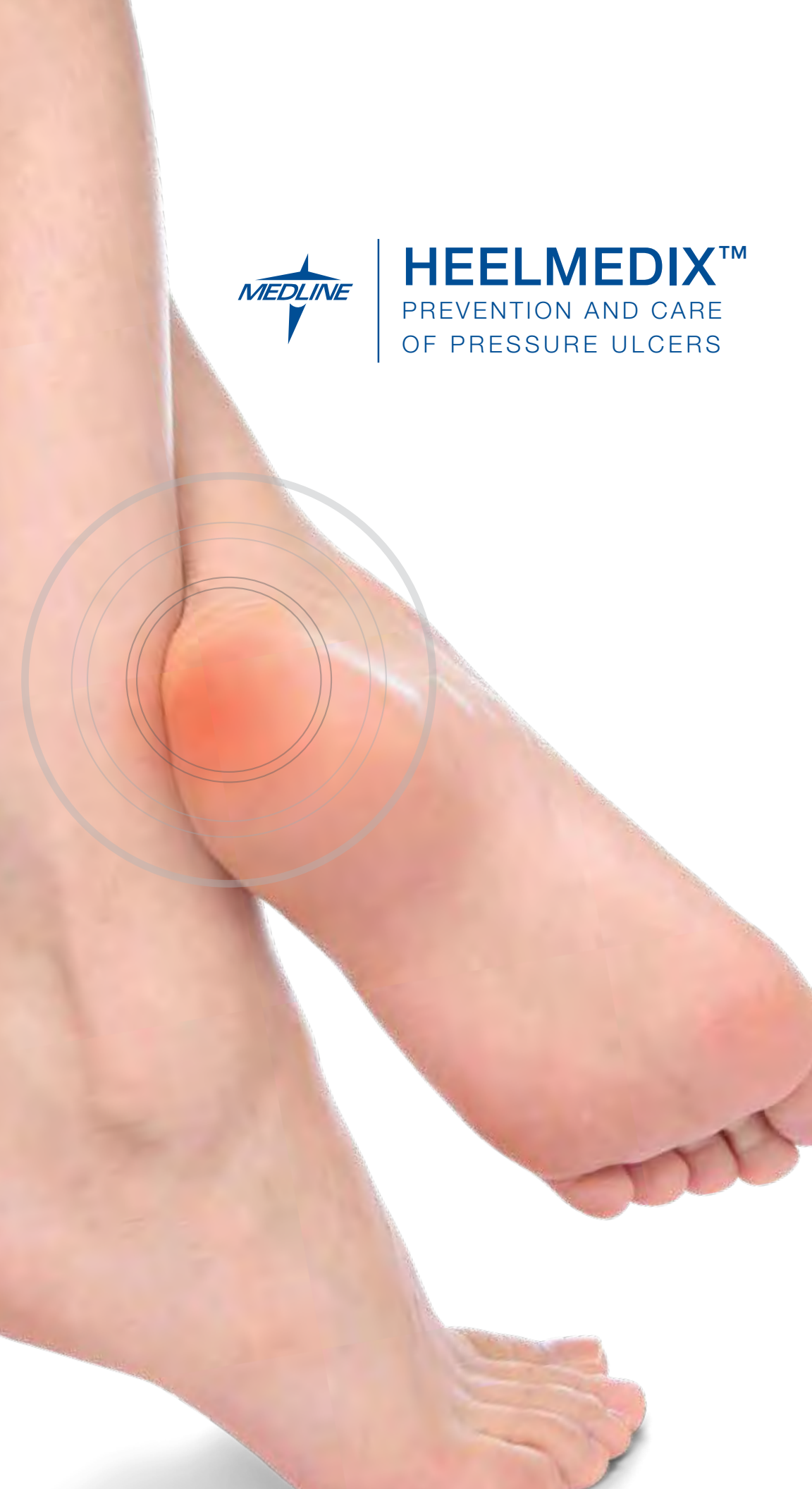

# HEELMEDIX™

PREVENTION AND CARE  
OF PRESSURE ULCERS



# Testing Shows HEELMEDIX Offers Superior Heel Off-Loading\*

## DESCRIPTION OF STUDY

Using a pressure-sensing mat, 19 subjects in a long-term care facility were tested for heel pressure on four surfaces: a mattress, pillow, in a HEELMEDIX device, and in the competitor's device.

## PRESSURE MAPPING CONCLUSIONS

### Mattress

The mattress presented the highest observed peak heel pressure.

### Hospital Pillow

The pillow reduced the heel pressure compared to the mattress alone.

### HEELMEDIX and Competitor

- Heelmedix completely off-loaded the vulnerable heel area
- Heelmedix exhibited the lowest average interface pressure across the entire contact area of the boot, compared to the competitor.



### MATTRESS



Average Pressure on Contact Area: **115.23**

### HOSPITAL PILLOW




Average Pressure on Contact Area: **64.40**

### HEELMEDIX




Average Pressure on Contact Area: **29.82**

### COMPETITOR

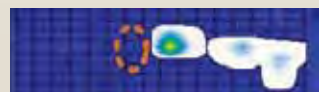

Average Pressure on Contact Area: **41.71**

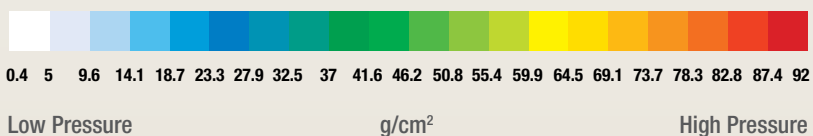

Contact your Medline representative about conducting a no-obligation trial in your facility.

\*Data on file



# HEELMEDIX

RELIEVE PRESSURE ON VULNERABLE HEELS

## Clinical Benefits

- Completely off-loads the heel and helps redistribute pressure
- Open heel design allows for airflow and easy monitoring
- Inner lining helps reduce friction and shear on heels — **half the friction of the competitor\***
- Inner material is absorbent, keeping the skin dry
- Helps protect against foot drop

### PILLOW-LIKE DESIGN

Provides comfort and protection

### UNIQUE STRAPS

Multiple strapping options dependent upon the patient needs

### STABILIZING WEDGES

Optional wedges can be used to help prevent medial & lateral rotation

\*Data on file

## MULTIPLE STRAPPING METHODS FOR CUSTOM FIT, COMFORT, AND RESULTS



### Foot Drop Strapping

For enhanced foot drop protection. Pressure, friction and shear are reduced within the area.



### Criss-Cross Strapping

Firmly isolates the foot while floating the heel. Pressure, friction and shear are reduced within the area.




### MESH LAUNDRY BAG INCLUDED

Simplifies washing and sorting



### COLOR-CODED TAGS

For quick size identification

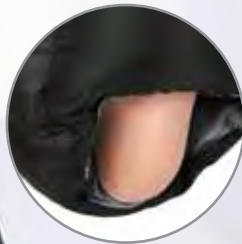

### OPEN HEEL DESIGN

For pressure off-loading and easy clinical checks



### OPENING ON EACH SIDE OF BOOT


Allows for DVT pressure devices

### NYLEX™ MATERIAL

Wedge is covered in nyllex material for easy cleaning

## Available in Three Sizes to Accommodate All Patients

To fit HEELMEDIX properly, measure the circumference of the patient's calf at its largest point. Color-coded tags (shown below) help to quickly identify size.

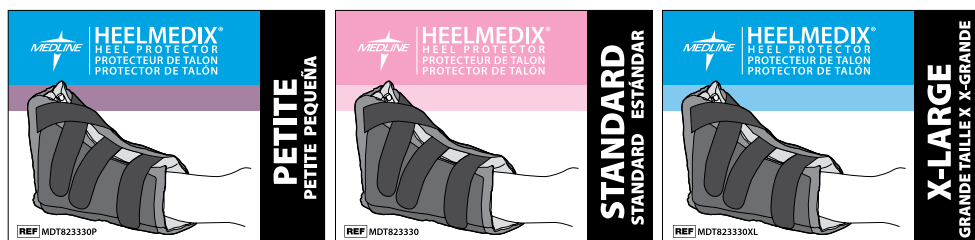

PETITE	STANDARD	X-LARGE
5-11 INCHES calf circumference (12.7-27.9 cm)	11-19 INCHES calf circumference (27.9-48.2 cm)	19-25 INCHES calf circumference (48.2-63.5 cm)

## Ordering Information

ITEM NUMBER	DESCRIPTION	PKG
MDT823330P	HEELMEDIX Heel Protector, Petite	1/ea
MDT823330SW	HEELMEDIX Heel Protector, Standard w/ Wedge	1/ea
MDT823330	HEELMEDIX Heel Protector, Standard	1/ea
MDT823330CS	HEELMEDIX Heel Protector, Standard	4/cs
MDT823330XL	HEELMEDIX Heel Protector, X-Large w/ Wedge	1/ea
MDT823330W	HEELMEDIX Stabilizer Wedge	2/ea
MDT823331W	HEELMEDIX Stabilizer Wedge	1/ea

## HEELMEDIX is part of Medline's Pressure Ulcer Prevention Program

Medline's Pressure Ulcer Prevention Program provides clinical and educational training resources for CNAs, RNs, LPNs, patients and their families. Administration tools and a program manager from Medline help you track results. More information is available at [medline.com/programs/pressure-ulcer-prevention-program](http://medline.com/programs/pressure-ulcer-prevention-program)



# Heel Raiser™

## Protect and suspend the heel.

The Heel Raiser provides total heel suspension. Fits left or right leg.

- One-size fits most
- Top folds down to accommodate small feet or compromised toes
- Anatomical strapping pattern reduces migration and foot drop



Top folds down

Open heel

ITEM NUMBER	DESCRIPTION	PKG
MDT823296	Heel Raiser Heel Protector, One Size Fits Most, Universal	1/ea

### VIDEOS AND MORE AT HEELMEDIX.COM:

- 1 Launch a QR Code reader app
- 2 Hover over this QR code
- 3 Or visit heelmedix.com



**Medline Industries, Inc.**  
One Medline Place  
Mundelein, IL 60060

**Medline United States**  
**1-800-MEDLINE (633-5463)**  
www.medline.com  
info@medline.com

**Medline Canada**  
**1-800-396-6996**  
www.medline.ca  
canada@medline.com

**Medline México**  
**01-800-831-0898**  
www.medlinemexico.com  
mexico@medline.com

Some products may not be available for sale in Mexico or Canada. We reserve the right to correct any errors that may occur within this brochure. ©2011 Medline Industries, Inc. Nylex is a trademark of Nylex Corporation, Ltd. HEELMEDIX and Heel Raiser are trademarks and Medline is a registered trademark of Medline Industries, Inc.