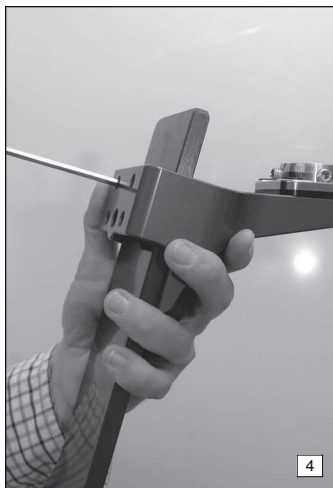
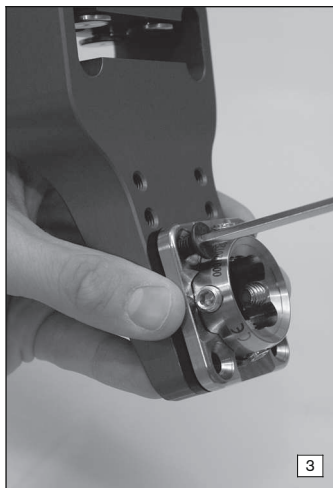
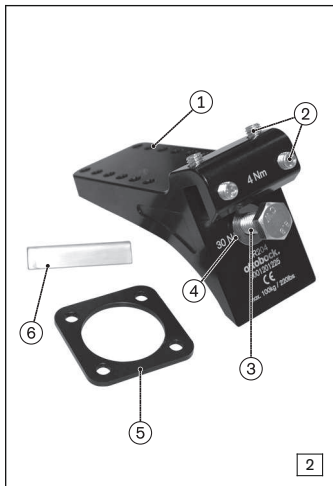
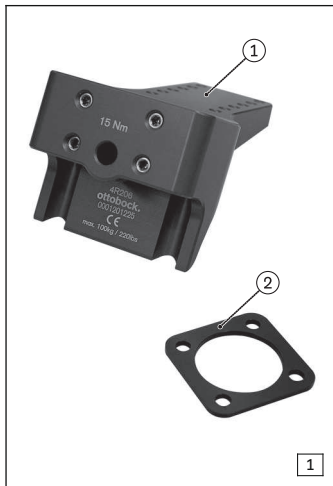
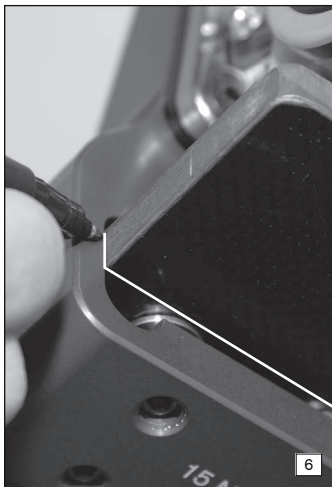
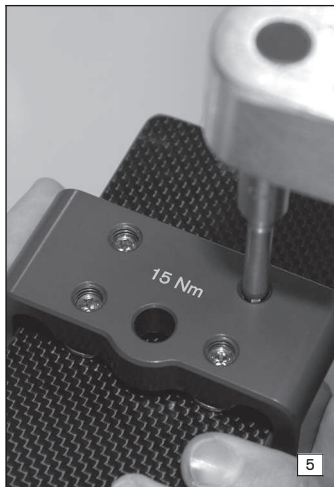
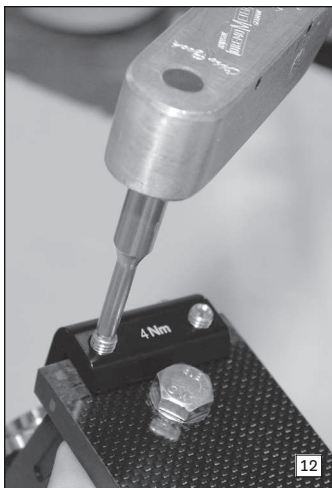
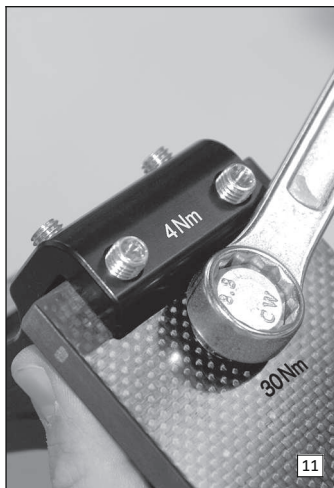
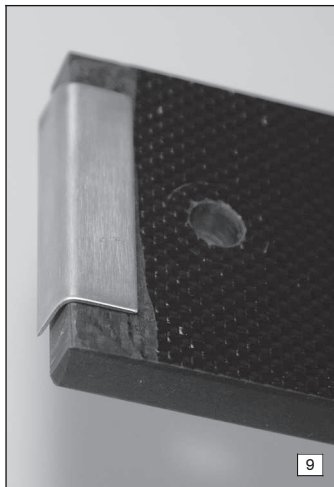
4R204, 4R206

DE Gebrauchsanweisung	5
EN Instructions for Use	12
FR Instructions d'utilisation	19
IT Istruzioni per l'uso	27
ES Instrucciones de uso	35
PT Manual de utilização	43
NL Gebruiksaanwijzing	50
SV Bruksanvisning	58
DA Brugsanvisning	65
NO Bruksanvisning	73
FI Käyttöohje	80
HR Upute za uporabu	87
JA 取扱説明書	94

1 Produktbeschreibung

INFORMATION

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INFORMATION

Date of the last update: 2013-03-11

- ▶ Please read this document carefully.
- ▶ Follow the safety instructions.

1.1 Construction and Function

The 4R204 and 4R206 TF sport foot adapters are mounted on the 1E90
Sprinter prosthetic foot.

- **Selecting the prosthetic foot:** TF test sport foot adapter **4R206**
- **Definitive sport prosthesis:** TF definitive sport foot adapter **4R204**

To improve the readability of the texts, the TF test sport foot adapter is
referred to as the test adapter below. The name definitive adapter is used for
the TF definitive sport foot adapter.

The prosthetic foot and prosthetic sport knee joint are joined by the TF sport
foot adapter with the help of a suitable socket adapter. The TF sport foot
adapter permits anterior or posterior placement of the prosthetic foot.

The **4R206 test adapter** is used for the selection of a suitable prosthetic foot. The optimum position and height of the prosthetic foot in the prosthesis are determined with the help of the clamp. The clamp can be reopened as often as desired.

When the definitive sport prosthesis is fabricated, the prosthetic foot is adapted and the test adapter is replaced by the **4R204 definitive adapter**.

1.2 Combination Possibilities

Designation	Reference Number
Socket Adapter	4R51, 4R54, 4R55, 4R73=A, 4R73=D, 4R77
Prosthetic Feet	1E90

For additional combination possibilities, please see the catalogue 646K2* or consult the manufacturer.

2 Application

2.1 Intended Use

The product is intended solely for lower limb prosthetic fittings.

2.2 Area of Application

Approved for a body weight of **up to 100 kg (220 lbs)**.

The 4R206 test adapter and the 4R204 definitive adapter may only be used for the fabrication of TF sport prostheses.

2.3 Environmental Conditions

Allowable Environmental Conditions
Temperature range for use: -10 °C to +60 °C (14 °F to 140 °F)
Allowable relative humidity 0 % to 90 %, non-condensing

Unallowable environmental conditions
Mechanical vibrations or impacts
Perspiration, urine, fresh water, salt water, acids
Dust, sand, highly hygroscopic particles (e. g. talcum)

2.4 Service Life

Based on ISO 10328, the prosthesis components were tested by the manufacturer for 3 million load cycles. With normal patient activity, this would correspond to a service life of 3 to 5 years in an everyday prosthesis. A general service life for this prosthesis component as part of a sport prosthesis cannot be determined, since strain varies widely depending on the sport and intensity.

3 Safety

3.1 Explanation of warning symbols

 **CAUTION** Warnings regarding possible risks of accident or injury.

 **NOTICE** Warnings regarding possible technical damage.

3.2 General Safety Instructions

 **CAUTION**

Using the product without following the instructions for use

Deteriorating health condition and damage to the product because of failure to follow the safety information

- ▶ Observe the safety information in these instructions for use.
- ▶ Make the patient aware of all safety instructions marked with "**Inform the patient**".

 **CAUTION**

Excessive strain on the product

Fall due to breakage of load-bearing components

- ▶ Use the prosthesis components according to the classification (see section "Area of Application").
- ▶ **Inform the patient.**

 **CAUTION**

Unallowable combination of prosthesis components

Fall due to breakage or deformation of the product

- ▶ Only combine the product with prosthesis components that are approved according to the section "Combination Possibilities".
- ▶ Based on the instructions for use of the prosthesis components, verify that they may be combined with each other.

 **CAUTION**

Improper assembly of the screw connections

Fall due to breakage or loosening of the screw connections

- ▶ Clean the threads before every installation.
- ▶ Apply the installation torques provided.
- ▶ Observe the instructions for securing the screw connections and the use of the correct length.

⚠ CAUTION**Exceeding the service life**

Fall due to change in or loss of functionality and damage to the product

- ▶ Ensure that the approved service life is not exceeded (see section "Service Life").
- ▶ **Inform the patient.**

NOTICE**Use under unallowable environmental conditions**

Damage to product due to unallowable environmental conditions.

- ▶ Do not expose the product to unallowable environmental conditions (see section "Environmental Conditions").
- ▶ If the product has been exposed to unallowable environmental conditions check it for damage.
- ▶ If damage is apparent or in case of doubt, do not continue using the product.
- ▶ Take suitable measures if required (e. g. cleaning, repair, replacement, inspection by the manufacturer or a specialist workshop etc.).
- ▶ **Inform the patient.**

Signs of changes in or loss of functionality during use

Among other factors, changes in functionality can be indicated by an altered gait pattern, a change in the positioning of the prosthesis components relative to each other and by the development of noises.

4 Scope of Delivery

The scope of delivery for the product is shown on page 2 (Figures 1 and 2).

The following single components and accessories are included in the scope of delivery in the specified quantities and are available for reordering as single components (■), single components with minimum order quantities (▲) or single-component packs (●):

4R206 TF test sport foot adapter					
Fig.	Item No.		Quantity	Designation	Reference Number
-	-	■	1	Instructions for Use	647G839
1	1		1	TF test sport foot adapter	-
1	2	■	1	Spacer Plate 3 mm	4G791
-	-	▲	4	Set Screw	4X298
-	-	▲	4	Countersunk Head Screw	501S41=M6x20

4R204 TF definitive sport foot adapter					
Fig.	Item No.		Quantity	Designation	Reference Number
–	–	■	1	Instructions for Use	647G839
2	1		1	TF definitive sport foot adapter	–
2	2	▲	4	Set Screw	506G3=M8x10
2	3	▲	1	Hexagon Head Screw	501K7=M10x30
2	4	▲	1	Flat Disc with Bevel	507U1=10
2	5	■	1	Spacer Plate 3 mm	4G791
2	6	■	1	Angle Plate	4X297
–	–	▲	4	Countersunk Head Screw	501S41=M6x20

5 Preparation for Use

⚠ CAUTION

Incorrect alignment or assembly

Injuries due to fall resulting from damaged prosthesis components

- Observe the alignment and assembly instructions.

INFORMATION

Not all of the materials listed may be available in your country.

- If the materials are not available, please contact the local branch of the manufacturer to obtain information on alternative materials.

5.1 Installing the Socket Adapter on the Test Adapter or Definitive Adapter

INFORMATION

Positioning the socket adapter

The socket adapter can be positioned on the test adapter or the definitive adapter in three different positions (offset **12 mm**).

Required tools and materials:

710D4 torque wrench, socket adapter, 4G791 spacer plate, 501S41=M6x20 countersunk head screws, degreasing cleaner (e.g. 636A58), 636K13 Loctite® 241

- 1) For definitive mounting, clean screw connection area using a degreasing cleaner.
- 2) Set the socket adapter with the intermediate pressure plate and the spacer plate onto the sport foot adapter in the desired position (see Fig. 3).

- 3) Secure the screw connections with Loctite.
- 4) Mount the socket adapter on the sport foot adapter with the countersunk head screws (**torque: 12 Nm**).

5.2 Installing the Prosthetic Foot in the Test Adapter

> Required tools and materials:

710D4 Torque Wrench

- 1) Slide the prosthetic foot into the adapter from the distal direction and move it to the desired position (see Fig. 4).
- 2) Tighten the 4 set screws to secure the prosthetic foot (see Fig. 5) (torque: **15 Nm**).

5.3 Shifting the Prosthetic Foot

> Required tools and materials:

710D4 torque wrench

> No load on the prosthesis.

- 1) Loosen the 4 set screws (see Fig. 4).
- 2) Shift the prosthetic foot in the test adapter.
- 3) Secure the prosthetic foot in the test adapter by tightening the 4 set screws (see Fig. 5) (torque: **15 Nm**).

5.4 Marking and Processing the Prosthetic Foot

After determining the definitive prosthetic foot height, the position of the prosthetic foot is marked on the test adapter and the prosthetic foot is processed accordingly.

> Required tools and materials:

Saw with metal or diamond blade, carbide or diamond drill bit Ø 10 mm

- 1) Mark the cutting edge of the prosthetic foot with a pen (see Fig. 6).
- 2) Mark the position of the bore hole with a pen (see Fig. 7).
- 3) Remove the test adapter from the prosthetic foot.
- 4) Clamp the prosthetic foot in a vice with jaw pads.
- 5) Saw through the prosthetic foot at the marking (see Fig. 8) and deburr the cut edge.
- 6) Drill the hole at the marked position and deburr it.

5.5 Installing the Prosthetic Foot in the Definitive Adapter

> Required tools and materials:

- Degreasing cleaner (e.g. 636A58), 636K13 Loctite® 241, 710D4 torque wrench
- > The prosthetic foot is shortened to the definitive length and has a bore hole.
- > The socket adapter is mounted on the definitive adapter (see Page 16)
- 1) For definitive mounting, clean screw connection area using a degreasing cleaner.
- 2) Turn out the set screws of the definitive adapter so that they do not protrude into the insertion space.
- 3) Set the angle plate onto the prosthetic foot (see Fig. 9) and slide the prosthetic foot into the clamping space up to the stop (see Fig. 10).
- 4) Face the bevel on the washer toward the screw head and slide the washer onto the hexagon head screw.
- 5) Secure the screw connections with Loctite.
- 6) Slide the hexagon head screw through the prosthetic foot into the definitive adapter and tighten it (see Fig. 11) (torque: **30 Nm**).
- 7) Tighten the 2 posterior set screws (see Fig. 12) (torque: **4 Nm**).
- 8) Tighten the 2 proximal set screws (torque: **4 Nm**).

6 Maintenance

CAUTION

Failure to follow the maintenance instructions

Injuries due to changes in or loss of functionality and damage to the product

- ▶ Observe the maintenance instructions.
- ▶ **Inform the patient.**

- During the consultation, listen for unusual noises and inspect the entire prosthesis for damage, wear and functional changes.
- Arrange regular maintenance intervals with the patient depending on the level of use.

Direct the patient to observe the following maintenance instructions:

- Check the entire prosthesis for damage after each use.
- In case of changes in or loss of functionality and damage, do not continue using the prosthesis and have it checked by qualified personnel.
- Comply with the maintenance intervals.

7 Legal Information

7.1 Liability

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the Instructions for Use. The device must only be operated with tested modular components. The manufacturer is not liable for damage caused by component combinations that were not authorised by the manufacturer.

7.2 CE Conformity

This device meets the requirements of the 93 / 42 / EEC guidelines for medical devices. This device has been classified as a class I device according to the classification criteria outlined in appendix IX of the guidelines. The declaration of conformity was therefore created by the manufacturer with sole responsibility according to appendix VII of the guidelines.

8 Technical Data

Reference Number	4R204	4R206
Weight [g]	440	580
Overall height [mm]	68	
System height [mm]	2	
Material	Aluminium	
Max. body weight [kg]	100	

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Otto Bock HealthCare GmbH