
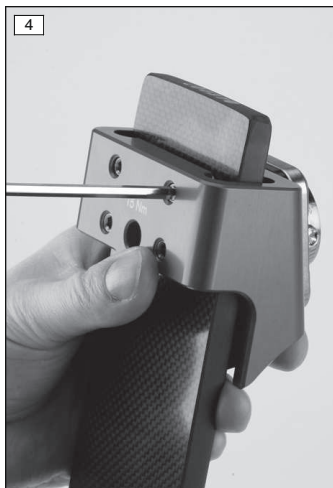
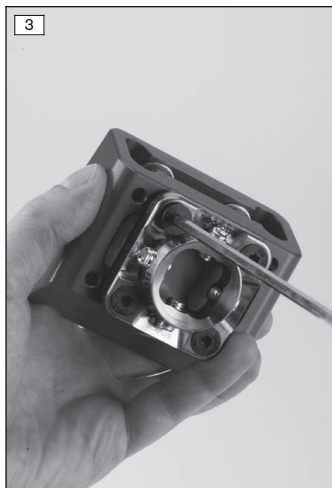
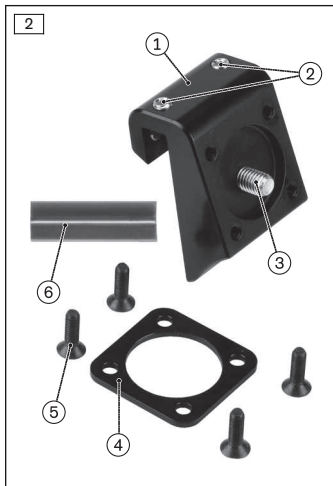
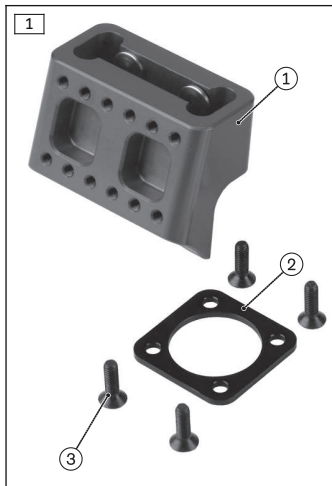
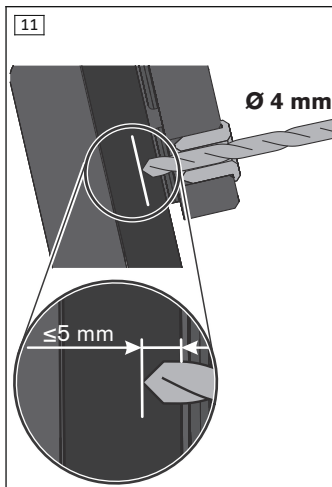
## 4R208, 4R210
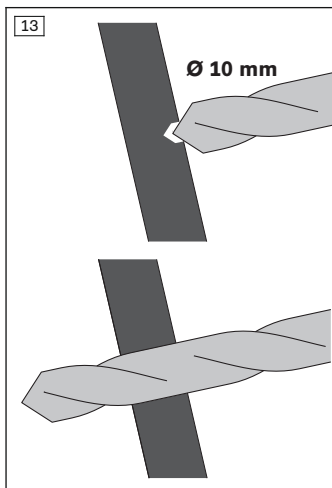
<b>DE</b>	Gebrauchsanweisung .....	6
<b>EN</b>	Instructions for Use .....	20
<b>FR</b>	Instructions d'utilisation .....	34
<b>IT</b>	Istruzioni per l'uso .....	49
<b>ES</b>	Instrucciones de uso .....	63
<b>PT</b>	Manual de utilização .....	78
<b>NL</b>	Gebruiksaanwijzing .....	92
<b>SV</b>	Bruksanvisning .....	107
<b>DA</b>	Brugsanvisning .....	120
<b>NO</b>	Bruksanvisning .....	134
<b>FI</b>	Käyttöohje .....	147
<b>HR</b>	Upute za uporabu .....	161

## 1 Produktbeschreibung

### INFORMATION

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### INFORMATION

Last update: 2014-05-06

- ▶ Please read this document carefully.
- ▶ Follow the safety instructions.

### 1.1 Construction and Function

The 4R208 and 4R210 TT Sport Foot Adapters are mounted on the 1E90 Sprinter prosthetic foot.

- **Selecting the prosthetic foot:** 4R210 TT Test Sport Foot Adapter
- **Definitive sport prosthesis:** 4R208 TT Definitive Sport Foot Adapter

To improve the readability of the texts, the TT test sport foot adapter is referred to as the test adapter below. The name definitive adapter is used for the TT definitive sport foot adapter.

The prosthetic foot and prosthetic socket are joined by the TT sport foot adapter with the help of suitable adapters (socket adapter, double adapter, lamination anchor).

The **4R210 Test Adapter** is used exclusively for the selection of a suitable prosthetic foot. The optimum position and height of the prosthetic foot in the prosthesis are determined with the help of the clamp. The clamp can be reopened as often as desired.

When the definitive sport prosthesis is fabricated, the prosthetic foot is adapted and the test adapter is replaced by the **4R208 Definitive Adapter**.

### 1.2 Combination Possibilities

### INFORMATION

In the sport prosthesis, only use prosthetic components approved for the respective sport or a body weight of 150 kg.

Prosthetic components that are especially well suited for combination with the product are listed below.

Designation	Reference Number
Sealing sleeve	453A3
Prosthetic socket	TSWB socket (see Technical Information 2.1.8)
Valve for prosthetic socket	4R136=EL, 4R140
Lamination anchor	4R42, 4R111, 4R116,
Double adapter	4R72*, 4R84, 4R76, 4R78
Socket adapter	4R51, 4R77
Prosthetic feet	1E90
Outsole	2Z500, 2Z501

For additional combination possibilities, please see the catalogue 646K2\* or consult the manufacturer.

## 2 Application

### 2.1 Intended Use

The product is to be used solely for lower limb prosthetic fittings.

### 2.2 Area of Application

Approved for a body weight of **up to 100 kg**.

The 4R210 Test Adapter and the 4R208 Definitive Adapter may only be used for the fabrication of TT sport prostheses.

### 2.3 Environmental Conditions

Allowable environmental conditions
Temperature range for use: -10 °C to +60°C
Allowable relative humidity 0 % to 90 %, non-condensing
Unallowable environmental conditions
Mechanical vibrations or impacts
Perspiration, urine, fresh water, salt water, acids
Dust, sand, highly hygroscopic particles (e.g. talcum)

### 2.4 Service Life

Based on ISO 10328, the prosthetic components were tested by the manufacturer for 3 million load cycles. With mobility not involving sports activities, this would correspond to a service life of 3 to 5 years in an everyday prosthesis. A general service life for this prosthetic component as part of a sport prosthesis cannot be determined, since strain varies widely depending on the sport and intensity of use.

## 3 Safety

### 3.1 Explanation of Warning Symbols

 **CAUTION** Warning regarding possible risks of accident or injury.

 **NOTICE** Warnings regarding possible technical damage.

### 3.2 General Safety Instructions

 **CAUTION**

**Using the product without following the instructions for use**

Deteriorating health condition and damage to the product because of failure to follow the safety information

- ▶ Observe the safety information in these instructions for use.
- ▶ Make the patient aware of all safety instructions marked with "**Inform the patient**".

 **CAUTION**

**Excessive strain on the product**

Fall due to breakage of load-bearing components

- ▶ Use the prosthetic components according to the classification (see section "Area of Application").
- ▶ **Inform the patient.**

 **CAUTION**

**Unallowable combination of prosthesis components**

Fall due to breakage or deformation of the product

- ▶ Only combine the product with prosthesis components that are approved according to the section "Combination Possibilities".
- ▶ Based on the instructions for use of the prosthesis components, verify that they may be combined with each other.

 **CAUTION**

**Exceeding the service life**

Fall due to change in or loss of functionality and damage to the product

- ▶ Ensure that the approved service life is not exceeded (see section "Service Life").
- ▶ **Inform the patient.**

### **CAUTION**

#### **Mechanical damage to the product**

Injuries due to changes in or loss of functionality

- ▶ Use caution when working with the product.
- ▶ If the product is damaged, check it for proper function and readiness for use.
- ▶ In case of changes in or loss of functionality, do not continue using the product (see "Signs of changes in or loss of functionality during use" in this section).
- ▶ Take any necessary measures (e.g. repair, replacement, inspection by the manufacturer's customer service, etc.).
- ▶ **Inform the patient.**

### **NOTICE**

#### **Use under unallowable environmental conditions**

Damage to product due to unallowable environmental conditions

- ▶ Do not expose the product to unallowable environmental conditions (see the section "Environmental Conditions").
- ▶ If the product has been exposed to unallowable environmental conditions, check it for damage.
- ▶ If damage is apparent or in case of doubt, do not continue using the product.
- ▶ Take suitable measures if required (e. g. cleaning, repair, replacement, inspection by the manufacturer or a specialist workshop etc.).
- ▶ **Inform the patient.**

#### **Signs of changes in or loss of functionality during use**

Among other factors, changes in functionality can be indicated by an altered gait pattern, a change in the positioning of the prosthetic components relative to each other and by the development of noises.

## **4 Scope of Delivery**

The scope of delivery for the product is shown on page 2 (see Fig. 1 and 2). The following single components and accessories are included in the scope of delivery in the specified quantities and are available for reordering according to their respective symbols as single components (■), single components with minimum order quantities (▲) or single-component packs (●):

4R208 TT Definitive Sport Foot Adapter					
Fig.	Item no.		Quantity	Designation	Reference Number
-	-	■	1	Instructions for Use	647G840
-	-	■	1	Quick Reference Guide	647H543
2	1	-	1	TT Definitive Sport Foot Adapter	-
2	2	▲	4	Set Screw	506G3=M8x10
2	3	▲	1	Hexagon Head Screw	501K7=M10x30
2	-	▲	1	Flat Bevelled Washer	507U1=10
2	4	■	1	Spacer Plate 3 mm	4G791
2	5	▲	4	Countersunk Head Screw	501S41=M6x20
2	6	■	1	Angle Plate	4X297

4R210 TT Test Sport Foot Adapter					
Fig.	Item no.		Quantity	Designation	Reference Number
-	-	■	1	Instructions for Use	647G840
-	-	■	1	Quick Reference Guide	647H543
1	1	-	1	TT Test Sport Foot Adapter	-
1	2	■	1	Spacer Plate 3 mm	4G791
1	3	▲	4	Countersunk Head Screw	501S41=M6x20
-	-	▲	4	Set Screw	4X298

## 5 Preparation for Use

### 5.1 Information on Fabrication of a Prosthesis

#### CAUTION

##### **Incorrect alignment or assembly**

Injuries due to fall resulting from damaged prosthesis components

- ▶ Observe the alignment and assembly instructions.

#### CAUTION

##### **Improper assembly of the screw connections**

Fall due to breakage or loosening of the screw connections

- ▶ Clean the threads before every installation.
- ▶ Apply the specified installation torque values.
- ▶ Observe the instructions for securing the screw connections and the use of the correct length.

**⚠ CAUTION****Initial use of the prosthesis by the patient**

Fall due to lack of patient experience, incorrect alignment or incorrect adjustment of the prosthesis

- ▶ For the safety of the patient, use a suitable device (e.g. parallel bars, handrail) during initial standing and walking.
- ▶ Do not use devices with insufficient stability (e.g. crutches or walking sticks).

**INFORMATION**

Not all of the materials listed may be available in your country.

- ▶ If the materials are not available, please contact the local branch of the manufacturer to obtain information on alternative materials.

Fabricating a prosthesis that is individually tailored to the patient demands technical knowledge of orthopaedic technology. The alignment and adjustments may only be carried out by a prosthetist.

The alignment and adjustments are mainly completed in the following steps:

1. Bench alignment
2. Static alignment
3. Dynamic trial fitting

The supplied Quick Reference Guide contains important information for the fabrication of the prosthesis and adaptation to the patient.

These steps are first carried out with a test prosthesis in order to determine the best possible combination and positioning of the prosthetic components relative to each other. Once all settings have been tailored to the patient, the definitive prosthesis is produced. The same steps are repeated for this purpose.

In dynamic trial fitting, the patient must be given sufficient time to become familiar with the functions of the prosthesis, thereby learning safe use for everyday life.

**5.2 Bench Alignment**

<b>Bench Alignment Process</b>	
<b>+ = anterior placement / - = posterior placement (in relation to the alignment reference line)</b>	
<b>No.</b>	<b>see Fig. 5, see Fig. 6</b>
	Fabricate the prosthetic socket according to the Technical Information 2.1.8.
	Perform the alignment in the alignment apparatus (e.g. 743L200 L.A.S.A.R. Assembly or 743A200 PROS.A. Assembly) or as illustrated in the 647H543

<b>Bench Alignment Process</b>	
<b>+ = anterior placement / - = posterior placement (in relation to the alignment reference line)</b>	
<b>No.</b>	<b>see Fig. 5, see Fig. 6</b>
	Quick Reference Guide (section "Gluing the Lamination Anchor into the Socket" – see page 8). Use a laser (e.g. 743L20*, 743L5) for the alignment reference line.
	Select the stiffness version of the prosthetic foot according to the selection tables in the 647H543 Quick Reference Guide (section "Selection of Stiffness Version of the 1E90" – see page 6).
<b>①</b>	Install the sole on the prosthetic foot according to its instructions for use.
<b>②</b>	Install the socket adapter on the foot adapter (section "Installing the Socket Adapter on the Foot Adapter" – see Page 27).
<b>③</b>	Install the foot adapter on the prosthetic foot ( <b>test adapter:</b> section "Installing the Test Adapter on the Prosthetic Foot" – see Page 27; <b>definitive adapter:</b> section "Installing the Definitive Adapter on the Prosthetic Foot" – see Page 27)
<b>④</b>	Connect the socket adapter to the lamination anchor (using a double adapter if needed) according to their instructions for use.
	During fabrication of the definitive prosthesis, mount the definitive socket on the lamination anchor according to the section "Information for Positioning and Gluing the Definitive Socket to the Lamination Anchor" (see Page 29).
	During fabrication of the test prosthesis, mount the test socket on the lamination anchor according to the section "Information for Positioning and Gluing the Test Socket to the Lamination Anchor" (see Page 28). Here the following positioning instructions also have to be observed:
	Position the proximal edge of the test adapter <b>40 mm</b> below the medial tibial plateau.
	Position the alignment reference point ② of the prosthetic socket (medial tibial plateau) relative to the sole according to the MTP – floor measurement.
	Set the determined socket flexion angle <b><math>\beta</math></b> .
<b>⑤</b>	a–p position of the mark ③ on the sole relative to the alignment reference line ①: <b>+30 – 35 mm</b>
<b>⑥</b>	External foot rotation: <b>0°</b>
<b>⑦</b>	Course of the frontal alignment reference line ①: <ul style="list-style-type: none"> <li>• Lateral edge of the patella ④</li> <li>• Centre of the prosthetic foot</li> </ul>

## 5.2.1 Installing the Socket Adapter on the Foot Adapter

### INFORMATION

#### Positioning the Socket Adapter on the Test Adapter

The socket adapter can be installed on the test adapter in three different positions (**12 mm** lateral offset – centred – **12 mm** medial offset).

#### > Required tools and materials:

710D4 Torque Wrench, socket adapter, 4G791 Spacer Plate, 501S41=M6x20 Countersunk Head Screws, degreasing cleaner (e.g. 636A58), 636K13 Loctite® 241

- 1) For definitive mounting, clean the screw connection area using a degreasing cleaner.
- 2) Set the socket adapter with the intermediate pressure plate and the spacer plate onto the sport foot adapter in the desired position (see Fig. 3).
- 3) Secure the screw connections with Loctite.
- 4) **INFORMATION: During installation, first tighten the proximal and then the distal countersunk head screws.**

Mount the socket adapter on the sport foot adapter with the countersunk head screws (**mounting torque: 12 Nm**).

## 5.2.2 Installing the Test Adapter on the Prosthetic Foot

#### > Required tools and materials:

710D4 Torque Wrench

- 1) Slide the test adapter onto the prosthetic foot from the proximal direction, move it to the desired position and tighten the 4 set screws (see Fig. 4).
- 2) Tighten the 4 set screws to the required mounting torque value (**15 Nm**) to secure the test adapter for the dynamic trial fitting (see Fig. 9).

## 5.2.3 Installing the Definitive Adapter on the Prosthetic Foot

### INFORMATION

The definitive adapter is only installed on the prosthetic foot when the definitive prosthesis is being fabricated. In order to do so, the correct prosthetic foot has to be selected and the proper positioning of the prosthetic components relative to each other must have been established.

> **Required tools and materials:**

Saw with metal or diamond blade, carbide or diamond drill bit Ø 4 mm + Ø 10 mm, degreasing cleaner (e.g. 636A58), 636K13 Loctite® 241, 710D4 Torque Wrench, vice, jaw pads

- 1) Mark the cutting edge according to the proximal edge of the test adapter on the prosthetic foot with a pen (see Fig. 10).
- 2) Drill the hole (Ø 4 mm) (see Fig. 11).
- 3) Remove the test adapter from the prosthetic foot.
- 4) Clamp the prosthetic foot in a vice with jaw pads.
- 5) Saw through the prosthetic foot at the marking (see Fig. 12) and deburr the cut edge.
- 6) Drill the hole (Ø 10 mm) and deburr it (see Fig. 13).
- 7) Remove the socket adapter from the test adapter and install it on the definitive adapter (section "Installing the Socket Adapter on the Foot Adapter" – see Page 27).
- 8) For definitive mounting, clean the screw connection area using a degreasing cleaner.
- 9) Turn out the set screws of the definitive adapter so that they do not protrude into the insertion space.
- 10) Set the angle plate onto the prosthetic foot and slide the prosthetic foot into the clamping space of the definitive adapter up to the stop (see Fig. 14).
- 11) Face the bevel on the washer toward the screw head and slide the washer onto the hexagon head screw.
- 12) Secure the screw connections with Loctite.
- 13) Slide the hexagon head screw through the prosthetic foot into the definitive adapter and tighten it (see Fig. 15) (mounting torque: **30 Nm**).
- 14) Tighten the 2 posterior set screws (see Fig. 16) (mounting torque: **4 Nm**).
- 15) Tighten the 2 proximal set screws (mounting torque: **4 Nm**).

### **5.2.4 Information on Positioning and Gluing the Test Socket to the Lamination Anchor**

> **Required materials:**

Adhesive putty (e.g. 639A1=1 + 617H21\* + 617P37\*), 699G30=7.5\* Cellacast Xtra Rigid Bandages

- 1) During positioning, ensure that the arms of the lamination anchor contact the test socket.
- 2) If required, install the double adapter between the lamination anchor and socket adapter to comply with the positioning instructions and values.

- 3) **INFORMATION: Use a cloth to protect the test adapter and prosthetic foot against contamination by the adhesive putty** (see 647H543 – section "Gluing the Lamination Anchor to the Socket" on page 8).

Once the correct position has been established, glue the test socket to the lamination anchor with adhesive putty.

Cover the arms of the lamination anchor entirely with the adhesive putty.

Allow the adhesive putty to set.

- 4) Secure the arms of the lamination anchor by wrapping them with rigid bandages (at least **2** layers).

### 5.2.5 Information on Positioning and Laminating the Lamination Anchor in the Definitive Socket

- Note the positioning of the prosthetic components in the fully adjusted test prosthesis and transfer to the definitive prosthesis.

## 5.3 Static Alignment

<b>Static Alignment Process</b>	
<b>+ = anterior placement / - = posterior placement (in relation to the load line)</b>	
<b>No.</b>	<b>see Fig. 7, see Fig. 8</b>
	<b>Required equipment:</b> 743L100 L.A.S.A.R. Posture
<b>①</b>	To determine the load line ①, position the patient on the L.A.S.A.R. Posture as follows: <ul style="list-style-type: none"> <li>• Prosthetic foot on force plate (sufficient load: <b>&gt; 35%</b> body weight)</li> <li>• Other foot (with shoe) on height compensation plate</li> <li>• The tips of the shoe and prosthetic foot are in one line</li> </ul>
<b>②</b>	Optimise the static alignment solely by changing the plantar flexion. Adjustment is performed only with the distal and proximal set screws of the socket adapter on the foot adapter. External foot rotation: <b>0°</b> Foot flexion in m-l direction: <b>0°</b> Position of the mark ③ on the sole relative to the load line: <b>0 mm</b> Position of the compromise pivot point ② (Nietert) relative to the load line: <b>&gt;= -15 mm</b>

## 5.4 Dynamic Trial Fitting

### CAUTION

#### Adjusting the Settings

Fall due to incorrect or unfamiliar settings

- ▶ Only adapt the settings to the patient gradually.
- ▶ Explain the effects of the adjustments on the use of the prosthesis to the patient.

During dynamic trial fitting, the alignment and settings of the prosthesis are checked and adapted for optimum walking and running according to the needs and abilities of the patient.

The patient has to learn the safe use of the prosthesis through intensive training.

The subsections that follow describe activities to be performed in the course of the dynamic trial fitting to adapt the prosthesis to the patient.

#### 5.4.1 Checking the Stiffness and Length of the Prosthetic Foot

- 1) In the 647H543 Quick Reference Guide, note the section "Check of stiffness and length of the 1E90" (see page 9).
- 2) **CAUTION! When hopping, do not bend the knee too far so that the proximal end of the prosthetic foot does not contact and injure the thigh.**  
Have the patient hop on both legs between parallel bars.
- 3) Check the level pelvis position.
- 4) If required, adapt the foot length according to the section "Shifting the Prosthetic Foot" (see Page 30) to achieve the level pelvis position.
- 5) If the stiffness of the prosthetic foot is not optimal, replace the prosthetic foot with one of different stiffness and once again check the level pelvis position while hopping.

##### 5.4.1.1 Shifting the Prosthetic Foot

###### > Required tools and materials:

710D4 Torque Wrench

- 1) Loosen the 4 set screws (see Fig. 4).
- 2) Shift the prosthetic foot in the required direction in the test adapter.
- 3) Secure the prosthetic foot in the test adapter by tightening the 4 set screws (see Fig. 9) (mounting torque: **15 Nm**).

## 5.4.2 Trimming the Prosthetic Foot

### > Required tools and materials:

Saw with metal or diamond blade, vice, jaw pads

- 1) In the 647H543 Quick Reference Guide, observe the section "Cutting the 1E90" (see page 10).
- 2) Mark a cut edge on the prosthetic foot with a pen so that the proximal edge of the prosthetic foot is **2 cm** from the thigh during the swing phase while running (flexion).
- 3) Clamp the prosthetic foot in a vice with jaw pads.
- 4) Saw through the prosthetic foot at the marking and deburr the cut edge.

## 5.4.3 Exercises

- ▶ Perform the exercises described in the 647H543 Quick Reference Guide (section "Exercises" – see page 12).

## 5.4.4 Dynamic Optimisation

For the dynamic optimisation of the sport prosthesis, it is important that the prosthetic foot is set to the following values:

- External foot rotation: **0°**
- Foot flexion in m-l direction: **0°**

The objective of dynamic optimisation is to optimally adapt the positioning of the prosthetic foot to the gait patterns of the patient. The following adjustment possibilities are used here:

- Posterior or anterior placement of the prosthetic foot in the a – p direction
- Shifting the Prosthetic Foot in the Test Adapter

Adjustment possibility of the prosthetic foot	Effect	Application
Posterior placement of the prosthetic foot	Increasing springiness to the front	When the patient has the feeling of being slowed down, or wants a more sporty gait.
Anterior placement of the prosthetic foot	Reducing springiness to the front	When the patient has the feeling of falling forward or wants a less sporty gait.
Shifting the prosthetic foot in the test adapter in the proximal direction	Reducing springiness upwards	When springiness upwards is excessive.
Shifting the prosthetic foot in the test adapter in the distal direction	Increasing springiness upwards	When springiness upwards is not sufficient.

- 1) In the 647H543 Quick Reference Guide, note the section "Dynamic Optimisation" (see page 16).
- 2) **CAUTION! See the instructions for use of the prosthetic components for the mounting torque values.**  
Use the distal and proximal set screws on the socket adapter for posterior or anterior placement of the prosthetic foot.
- 3) To shift the prosthetic foot in the test adapter, see the section "Shifting the Prosthetic Foot" (see Page 30).

## 5.5 Finishing the Prosthesis

### > Required tools:

Torque wrench

- 1) Tighten all screws of the prosthetic components to the specified installation torque values.
- 2) Verify proper functioning of the prosthesis.

## 6 Use

### NOTICE

#### Mechanical overload

Impaired functionality due to mechanical damage

- ▶ Check the product for damage prior to each use.
- ▶ Do not use the product if its functionality has been impaired.
- ▶ Take any necessary measures (e.g. repair, replacement, inspection by the manufacturer's customer service, etc.).
- ▶ **Inform the patient.**

## 7 Maintenance

### CAUTION

#### Failure to follow the maintenance instructions

Injuries due to changes in or loss of functionality and damage to the product

- ▶ Observe the maintenance instructions.
  - ▶ **Inform the patient.**
- ▶ Arrange regular maintenance intervals with the patient depending on the level of use.
  - ▶ Following an individual period for the patient to get accustomed to the prosthesis, check the settings of the prosthetic joint and adapt them to the patient's requirements again as needed.

- ▶ The prosthetic components should be inspected after the first 30 days of use.
- ▶ Inspect the entire prosthesis for wear during normal consultations.
- ▶ Conduct annual safety inspections.
- ▶ Direct the patient to observe the following maintenance instructions:  
Check the entire prosthesis for damage after each use.  
In case of changes in or loss of functionality and damage, do not continue using the prosthesis and have it checked by qualified personnel.  
Comply with the maintenance intervals.

## 8 Disposal

This product may not be disposed of with regular domestic waste in all jurisdictions. Disposal that is not in accordance with the regulations of the country where the product is used may have a detrimental impact on health and the environment. Please observe the information provided by the responsible authorities in the country of use regarding return, collection and disposal procedures.

## 9 Legal Information

### 9.1 Liability

The manufacturer will only assume liability if the product is used in accordance with the descriptions and instructions provided in this document (Instructions for Use). The manufacturer will not assume liability for damage caused by disregard of this document.

### 9.2 CE Conformity

This device meets the requirements of the 93 / 42 / EEC guidelines for medical devices. This device has been classified as a class I device according to the classification criteria outlined in appendix IX of the guidelines. The declaration of conformity was therefore created by the manufacturer with sole responsibility according to appendix VII of the guidelines.

## 10 Technical data

Reference Number	4R208	4R210
Weight [g]	284	385
Material	Aluminium	
Max. body weight [kg]	100	

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Otto Bock HealthCare GmbH